


Seat regulation angles:

- A-a 6°
- A-b 4°
- A-c 2°
- A-d 0°

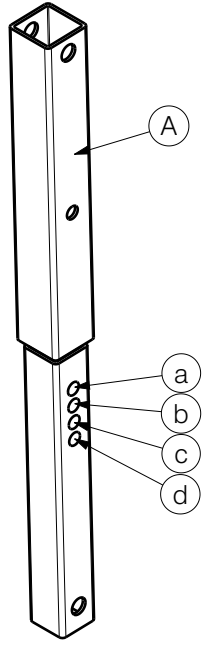

EP00470 / N

| | | | | |
|--|------------|------------|------------|------------|
| TURIOS BASIC BACKREST AND SEAT Sheet 1/7 | | | | Plan No. |
| | | | | EP00470/ N |
| Scale: 1:5 | Drafted | Verified | Confirmed | |
| Date | 26/11/2019 | 19/09/2023 | 19/09/2023 | |
| Initials | DB | PM | MK | |

| Material | | | |
|----------|-------------|---------|----------|
| No. | Plan | Article | Quantity |
| 1 | YP07428 (S) | 2778654 | 1 |
| | YP07429 (M) | 2778754 | 1 |
| | YP07430 (L) | 2778854 | 1 |
| 2 | YP07431 (S) | 2778954 | 1 |
| | YP07432 (M) | 2779054 | 1 |
| | YP07433 (L) | 2779154 | 1 |
| 3 | YP07434 (S) | 2779254 | 1 |
| | YP07435 (M) | 2779354 | 1 |
| | YP07436 (L) | 2779454 | 1 |
| 4 | YP07736 | 2781754 | 1 |
| | YP07737 | 2781854 | 1 |
| | YP07750 (S) | 2780854 | 1 |
| 6 | YP07751 (M) | 2780954 | 1 |
| | YP07752 (L) | 2781054 | 1 |
| | YP07753 (S) | 2781154 | 1 |
| 7 | YP07754 (M) | 2781254 | 1 |
| | YP07755 (L) | 2781354 | 1 |
| | YP07756 (S) | 2781454 | 1 |
| 8 | YP07757 (M) | 2781554 | 1 |
| | YP07758 (L) | 2781654 | 1 |
| | SAFETY BELT | 2993352 | 1 |
| 9 | M8x40 | 2994122 | 4 |
| 11 | XP02887 | GP21700 | 2 |
| 12* | X004942 | 1907377 | 2 |
| 13* | X004943 | 1907376 | 2 |
| 14 | Y012123 | 1675800 | 2 |
| 15 | YP08993 | 2631054 | 1 |
| 16 | YP08994 | 2631154 | 1 |

* - Lights SE19 - option

| Material | | | |
|----------|-------------------|---------|----------|
| No. | Plan | Article | Quantity |
| 17* | X004960 | 1907430 | 2 |
| 18* | --- | 1910745 | 2 |
| 19 | M6x30 Black | 1915416 | 8 |
| 20 | M6 DIN125A | 1001564 | 10 |
| 21 | M6 DIN985 | 2994063 | 20 |
| 22 | M6x45 | 1991017 | 4 |
| 23 | M6x16 | 1008459 | 2 |
| 24 | M10 DIN985 | 1001683 | 2 |
| 25 | M6x16 DIN912 | 1001492 | 2 |
| 26* | M5x16 DIN912 | 1007198 | 2 |
| 27* | M5x30 DIN912 | 1903983 | 2 |
| 28* | LedHeadLight | 1907340 | 2 |
| 29* | IndicatorLamp | 1991679 | 2 |
| 30 | M8x40 | 1008739 | 2 |
| 31 | M8 DIN985 | 1001682 | 6 |
| 32 | GN 717-8-AK | 2990455 | 2 |
| 33 | M8 DIN125 BLACK | 2994415 | 4 |
| 34 | M10 DIN125A | 1001558 | 4 |
| 35 | M10x50 | 1009335 | 1 |
| 36 | M10x60 | 1907128 | 1 |
| 37 | D8x45 | 1993594 | 1 |
| 38 | 6x40 | 1006111 | 4 |
| 39 | M5 DIN 985 | 2994006 | 10 |
| 40 | M5x35 | 2990115 | 2 |
| 41 | LE 50A (6km/h) | 1910896 | 1 |
| | LINX 60A (10km/h) | 1912294 | 1 |
| 42 | YP07971 | 2603654 | 1 |
| 43 | X005565 | 1912287 | 2 |
| 44 | M6 DIN985 | 2994063 | 4 |
| 45 | M6x40 | 1006850 | 4 |
| 46 | M5x20 DIN912 | 1915452 | 8 |
| 47 | Plastic Plug | 1001602 | 2 |

TURIOS BASIC BACKREST AND SEAT

Sheet 2/7

Plan No.

EP00470 / N

Scale: 1:5

Drafted

Verified

Confirmed

Date

26/11/2019

19/09/2023

19/09/2023

Initials


DB

PM

MK




EP00470 / N

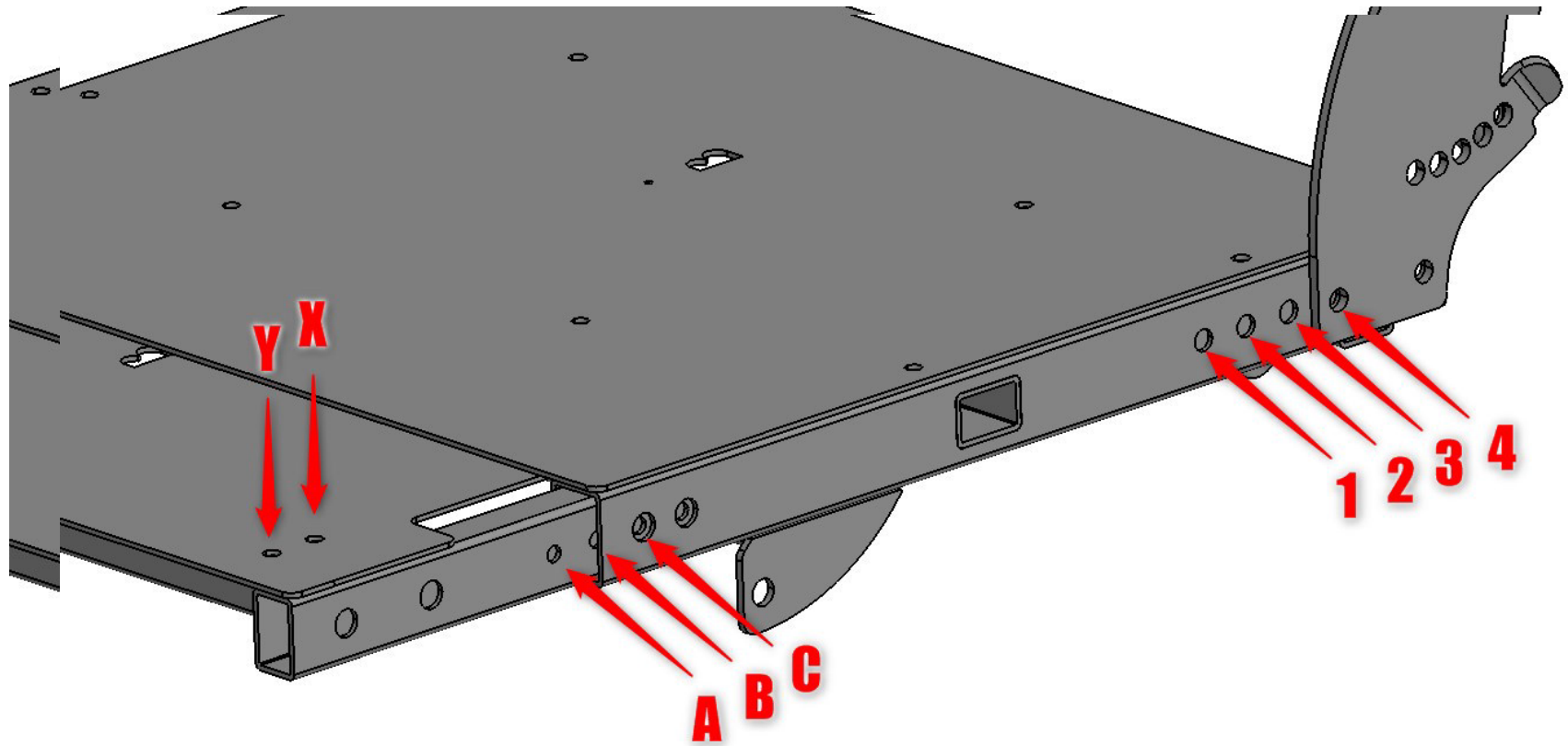


EP00470 / N

| | | | | |
|--|------------|------------|------------|-------------|
| <p>TURIOS BASIC BACKREST AND SEAT</p> <p>Sheet 3/7</p> | | | | Plan No. |
| | | | | EP00470 / N |
| Scale: 1:5 | Drafted | Verified | Confirmed | |
| Date | 26/11/2019 | 19/09/2023 | 19/09/2023 | |
| Initials | DB | PM | MK | |

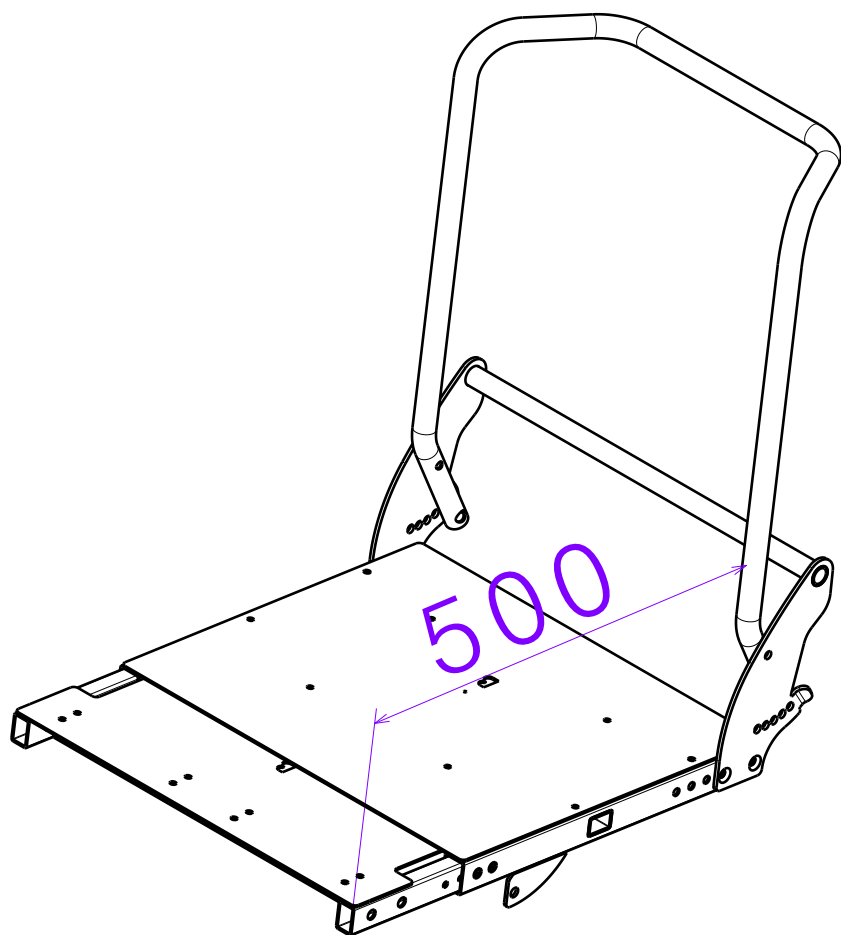


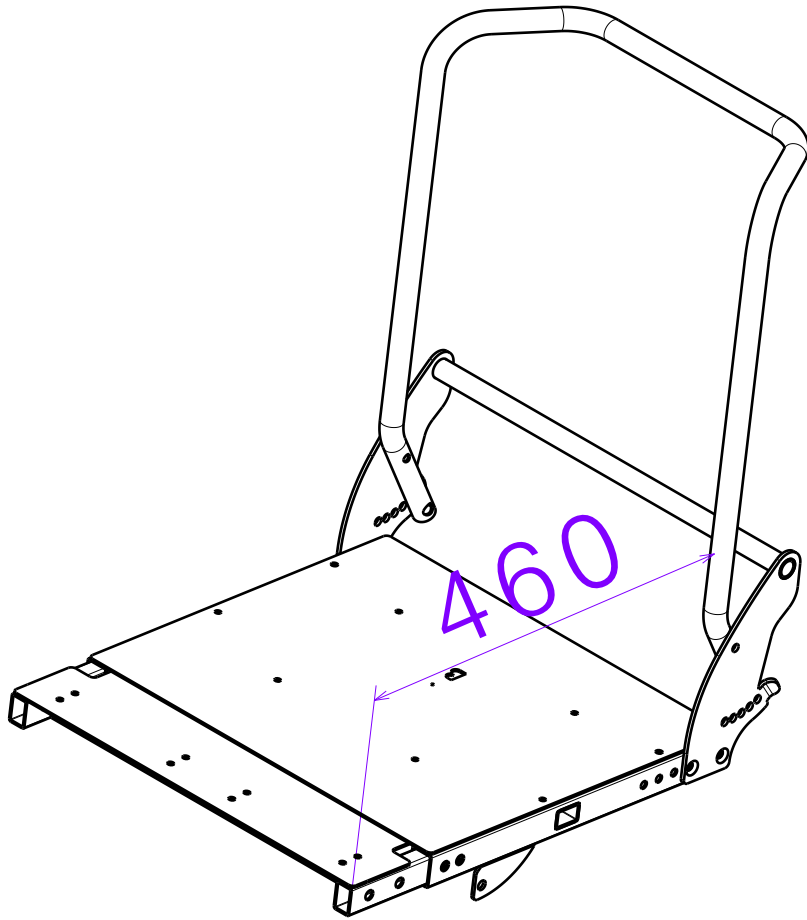
Size 500 / Rozmiar 500
XC4

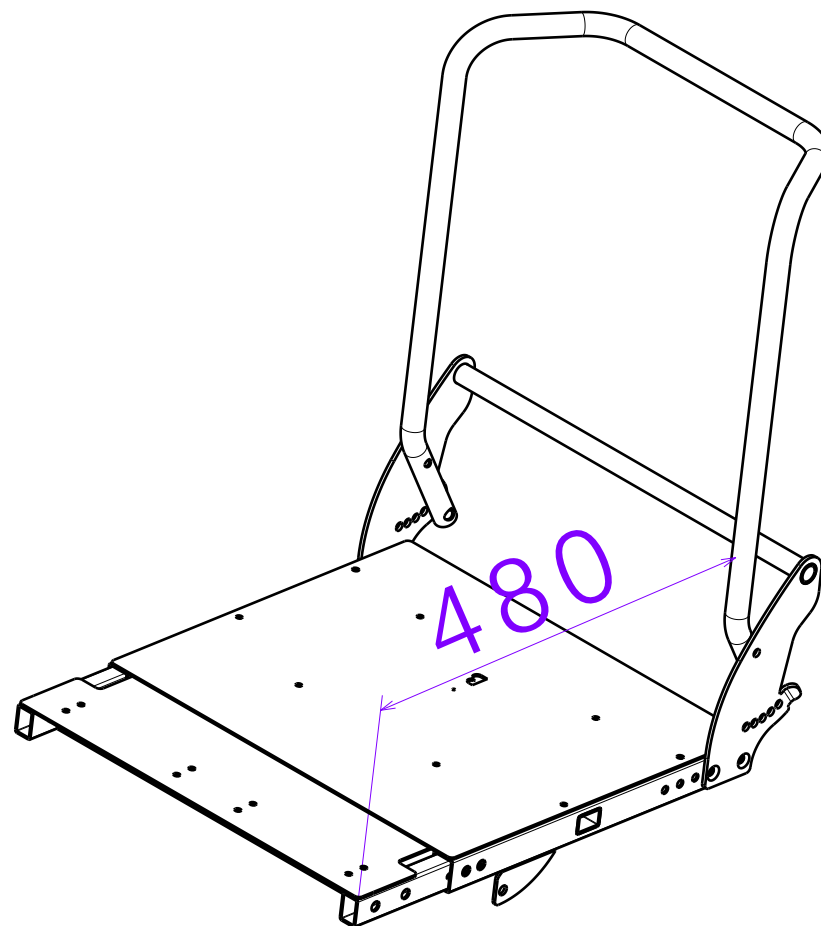


Size 480 / Rozmiar 480
XB4

EP00470 / N

| | | | | |
|--------------------------------|------------|------------|------------|---|
| TURIOS BASIC BACKREST AND SEAT | | | | Plan No. |
| | | | | EP00470 / N |
| Sheet 4/7 | | | |  |
| Scale: 1:5 | Drafted | Verified | Confirmed | |
| Date | 26/11/2019 | 19/09/2023 | 19/09/2023 | |
| Initials | DB | PM | MK | |




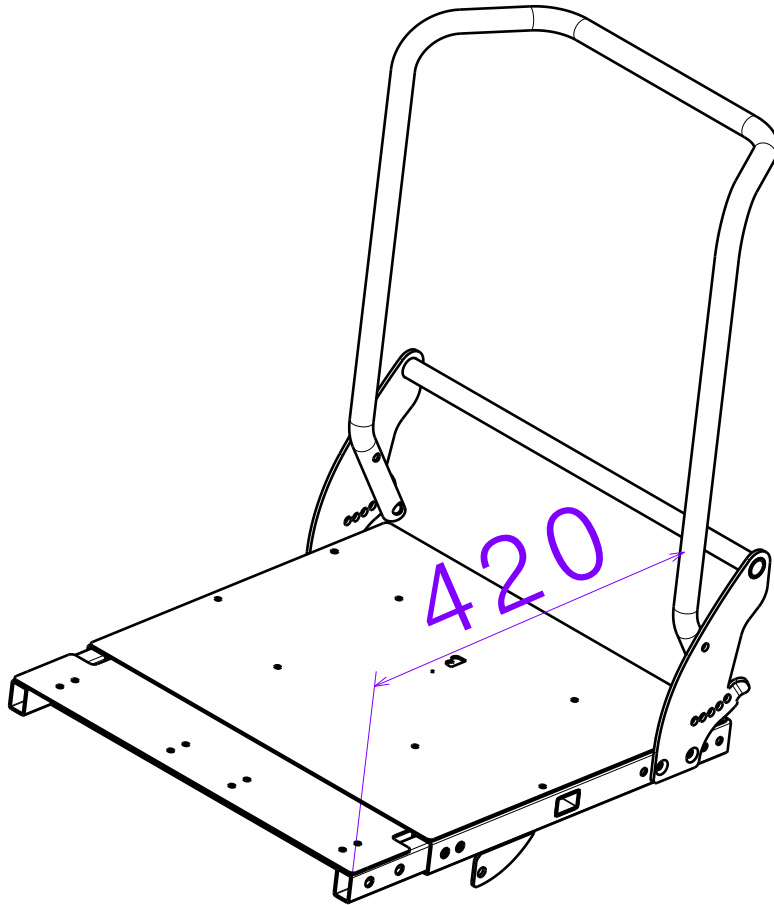
Size 460 / Rozmiar 460
XA4

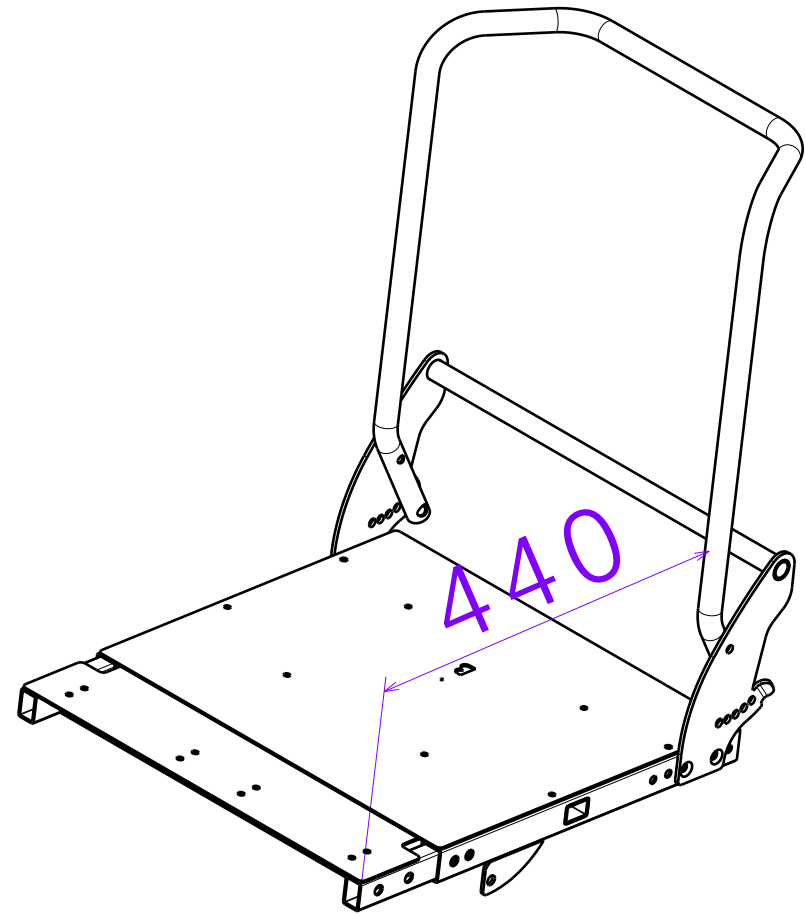


Size 440 / Rozmiar 440
XA3

EP00470 / N

| | | | | |
|--------------------------------|------------|------------|------------|---|
| TURIOS BASIC BACKREST AND SEAT | | | | Plan No. |
| | | | | EP00470 / N |
| Sheet 5/7 | | | |  |
| Scale: 1:5 | Drafted | Verified | Confirmed | |
| Date | 26/11/2019 | 19/09/2023 | 19/09/2023 | |
| Initials | DB | PM | MK | |




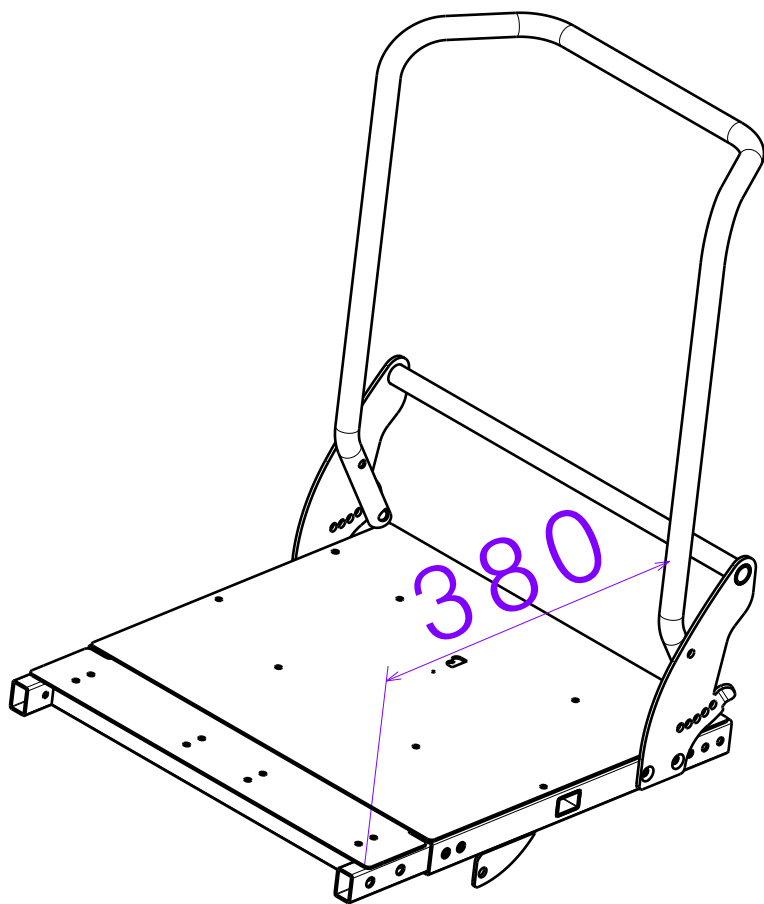
Size 420 / Rozmiar 420
XA2

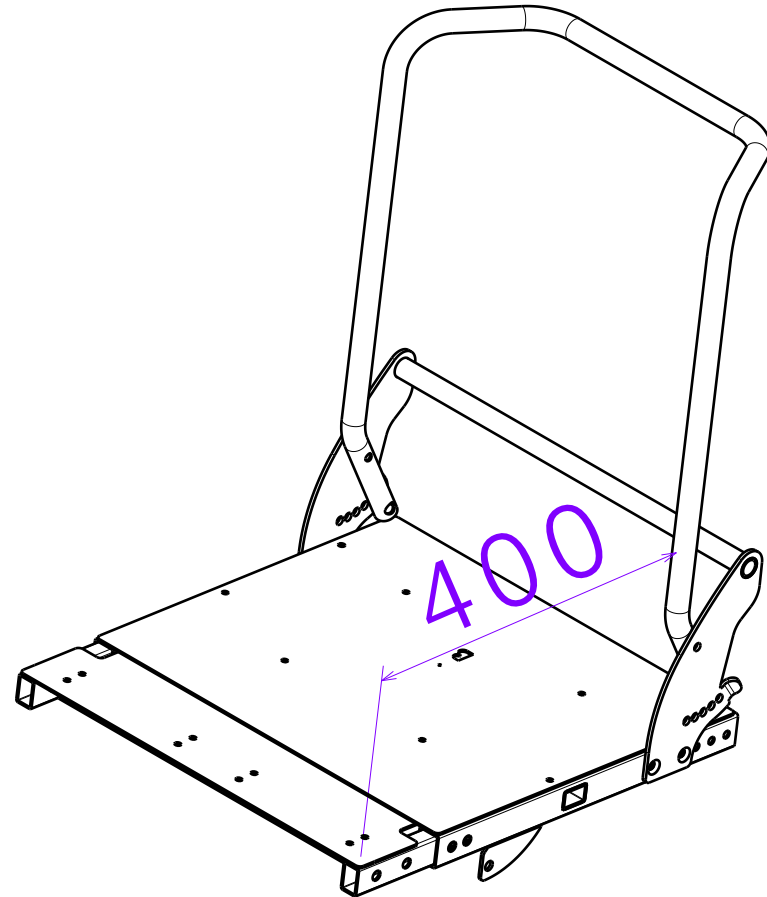

Size 400 / Rozmiar 400
XA1


EP00470 / N

| | | | | |
|--------------------------------|------------|------------|------------|---|
| TURIOS BASIC BACKREST AND SEAT | | | | Plan No. |
| | | | | Sheet 6/7 EP00470/N |
| Scale: 1:5 | Drafted | Verified | Confirmed |  |
| Date | 26/11/2019 | 19/09/2023 | 19/09/2023 | |
| Initials | DB | PM | MK | |



Size 380 / Rozmiar 380
YA1

| No. | Size | Configuration |
|-----|------|---------------|
| 1 | 380 | YA1 |
| 2 | 400 | XA1 |
| 3 | 420 | XA2 |
| 4 | 440 | XA3 |
| 5 | 460 | XA4 |
| 6 | 480 | XB4 |
| 7 | 500 | XC4 |

| | | | | |
|--|------------|------------|------------|---|
| TURIOS BASIC BACKREST AND SEAT Sheet 7/7 | | | | Plan No. |
| | | | | EP00470/ N |
| Scale: 1:5 | Drafted | Verified | Confirmed |  |
| Date | 26/11/2019 | 19/09/2023 | 19/09/2023 | |
| Initials | DB | PM | MK | |

EP00470 / N