



Material / Materiaal			
No. / Nr.	Plan	Article / Artikel	Quantity / Aantal
1	X001016	1001682	1
2	X002133	1992424	4
3	X003692	1998763	4
4	X004452	1904058	4
5	X004606	1905091(39/44/48)	2
	X004607	1905092(42/46/50)	2
6	X004620	1905090	2
7	Y015110	1728754 (39)	1
	Y015113	1728954 (42)	1
	Y015099	1729154 (44)	1
	Y015116	1729354 (46)	1
	Y015119	1729554 (48)	1
	Y015122	1729754 (50)	1
8	Y015111	1728654 (39)	1
	Y015114	1728854 (42)	1
	Y015100	1729054 (44)	1
	Y015117	1729254 (46)	1
	Y015120	1729454 (48)	1
	Y015123	1729654 (50)	1
9	3.9x9.5 DIN7981	1001530	4
10	X001119	1001554	2
11	M4 DIN985	1001678	4
12	M6 DIN985	1001680	6
13	M6x40 DIN912	1003909	4
14	M6x50 DIN912	1007970	2
15	M8x110 DIN912	1991711	1
16	---	2990185	4
18A	---	H001126	0.86
18	---	1905685 (39)	1
	---	1905686 (42)	1
	---	1905687 (44)	1
	---	1905688 (46)	1
	---	1905689 (48)	1
	---	1905690 (50)	1

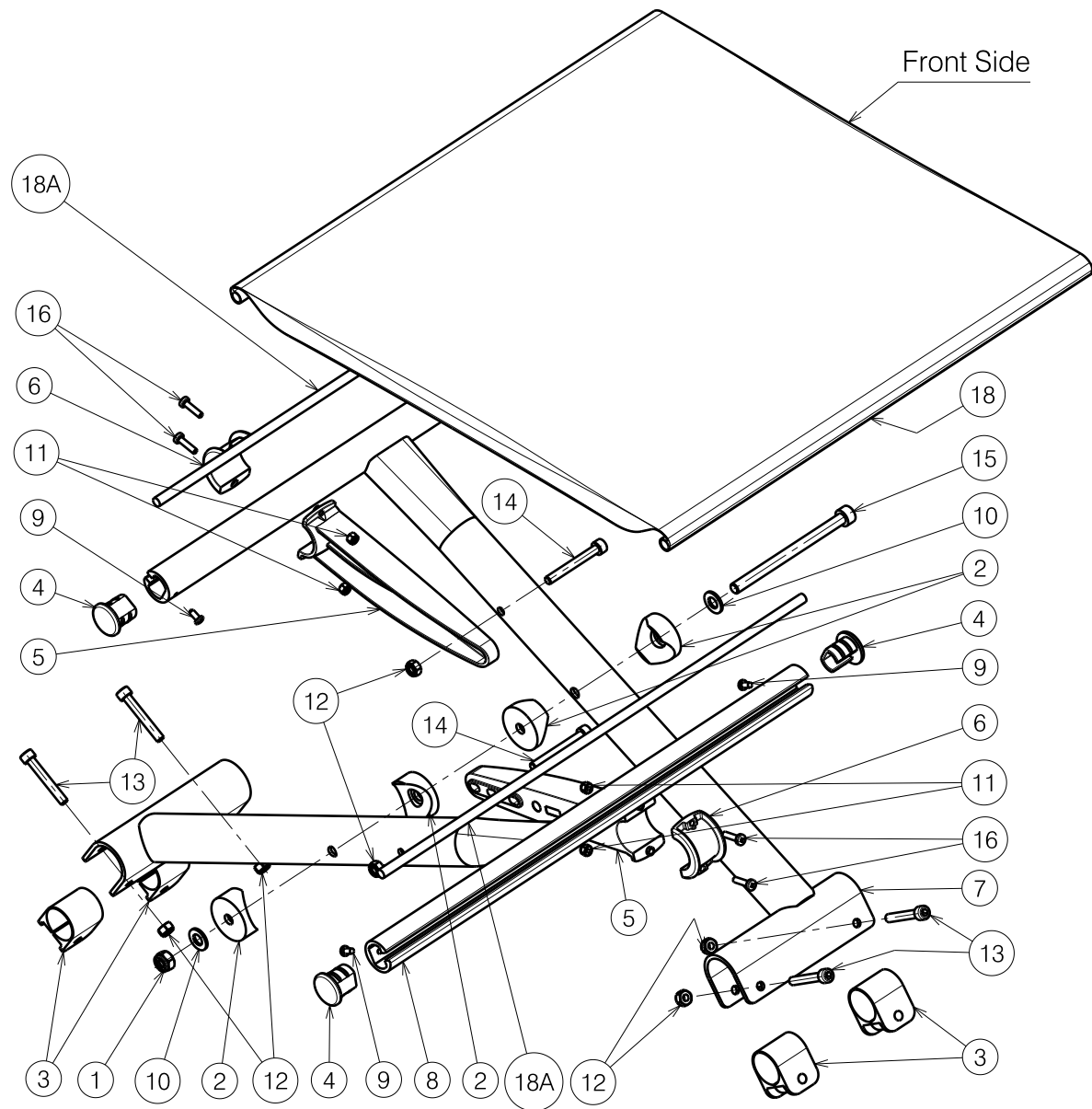

E000624 / H

CROSS V200 KRUIS V200				Plan No. / Plannr. E000624 / H
Scale / Schaal: 1:5	Drafted / Getekend	Verified / Nagezien	Confirmed / Bevestigd	
Date / Datum	26/04/2010	13/09/2018	13/09/2018	
Init. / Init.	DD	SD	PJ	