


Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)

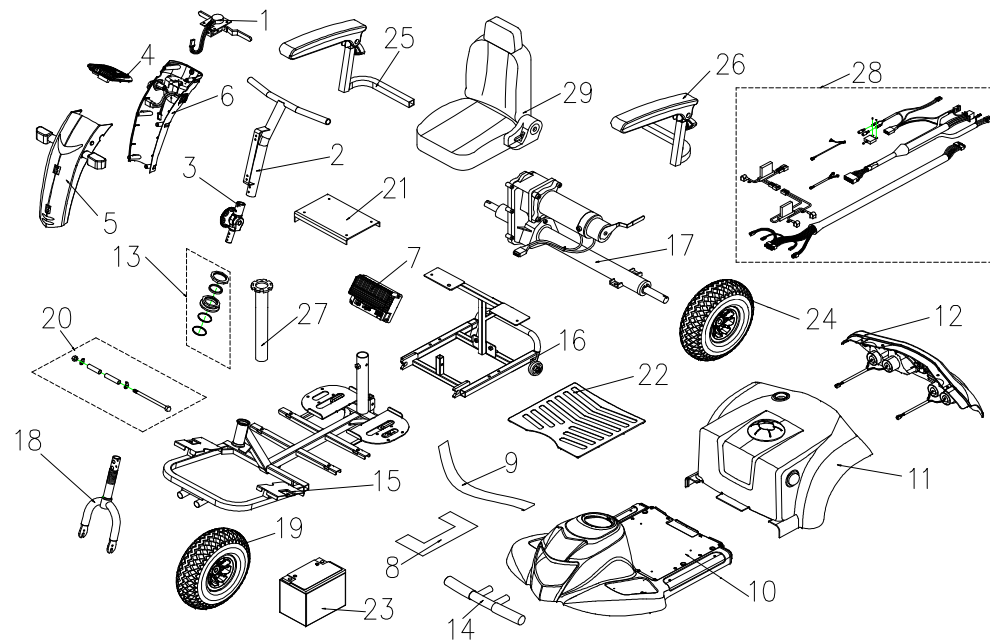

Ceres 3 (M3B)


Ceres 3 (M3B) Parts List (2009/05/20)				
Fig&Index NO.	Parts No.	Parts Name	QTY	Remark
		English		
1	S12-009-00801	SPEED DIAL ASS'Y	1set	1904367
2	S02-055-00100	T TYPE HANDLE BAR ASS'Y	1set	
3	S02-001-00501	TILLER SWIVEL ASS'Y	1set	
4	S08-054-00100	COVER , HANDLE BAR ASS'Y , UPPER	1set	
5	S08-055-00157	FRONT COVER ASS'Y , HAND BAR, BLUE 57 , W/O LABEL	1set	
5	S08-055-00166	FRONT COVER ASS'Y , HAND BAR, BLACK 66 , W/O LABEL	1set	
6	S08-055-00257	REAR COVER ASS'Y , HAND BAR , BLUE 57	1set	
6	S08-055-00266	REAR COVER ASS'Y , HAND BAR , BLACK 66	1set	
7	C10-001-01100	CONTROLLER , CTE 70A	1	
8	C25-003-00500	RUBBER PAD , BATTERY	1	
9	C07-001-90100	VELCRO STRIP , BATTERY	1	
10	S08-055-00357	FRONT SHROUD ASS'Y , BLUE 57	1set	
10	S08-055-00366	FRONT SHROUD ASS'Y , BLACK 66	1set	
11	S08-054-00557	COVER ASS'Y , BATTERY , BLUE 57 , W/O LABEL	1set	
11	S08-054-00566	COVER ASS'Y , BATTERY , BLACK 66 , W/O LABEL	1set	
12	S08-054-00657	REAR SHROUD ASS'Y , BLUE 57 , W/O LABEL	1set	
12	S08-054-00666	REAR SHROUD ASS'Y , BLACK 66 , W/O LABEL	1set	
13	S02-001-00601	REGULAR BEARING BALL SET	1set	

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)				
Fig&Index NO.	Parts No.	Parts Name	QTY	Remark
		English		
14	C01-055-00300	FRONT BUMPER	1	
15	S01-055-00100	FRAME,FRONT SECTION ASS'Y	1set	
16	S01-054-00200	FRAME,REAR SECTION ASS'Y	1set	
17	S09-054-00100	REAR DRIVE ASS'Y (W/I BOLT / NUT) ,9M/5300RPM/26:1	1	
18	C03-042-00101	FRONT FORK ASS'Y	1	
19	S05-055-00100	FRONT TIRE ASSY,300-4-C154-1 GRAY	1set	
20	S05-042-00100	HEX FLANGE BOLT , M10xP1.25x160L	1set	
21	S01-054-00300	HOLDER , CONTROLLER ASS'Y	1set	
22	C21-055-00100	CARPET	1	
23	C17-001-23402	BATTERY , U1	1	
24	S06-054-00100	REAR TIRE ASSY , GRAY , 400-5	1set	
25	S07-053-00300	ARMREST COMP. , R	1set	
26	S07-053-00100	ARMREST , ADJUSTABLE , L	1	
27	C07-023-00102	SEAT POST	1	
28	S18-054-00100	WIRE HARNESS ASS'Y	1set	
29	S07-055-00100	18" SEAT ASS'Y , BLACK , W/I SLIDER-100mm	1set	
30		ACCESSORIES		
31		LABEL ACCESSORIES		

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)

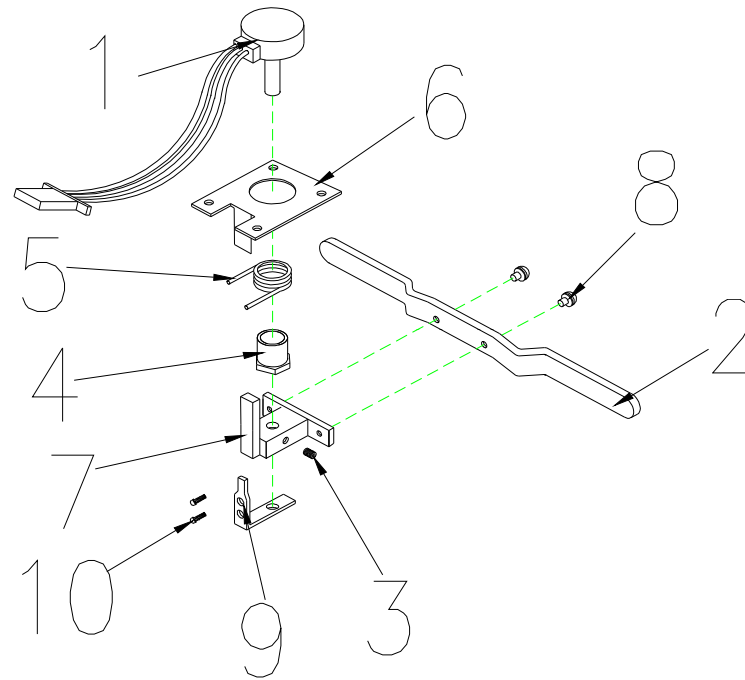


1. SPEED DIAL ASS'Y

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)					
Fig&Index NO.	Parts No.	Parts Name		QTY	Remark
		English			
1	S12-009-00801	SPEED DIAL ASS'Y		1set	1904367
1-1	C12-001-01401	SPEED LEVER COMP.		1	
1-2	C12-009-00600	SPEED LEVER		1	
1-3	9358040040753	LOCKING SCREW,4x4		2	
1-4	C12-001-00101	NUT		1	
1-5	C12-001-00300	TORSION SPRING,SPEED LEVER		1	
1-6	C12-001-00402	FIX PLATE,SPEED LEVER		1	
1-7	C12-009-00700	HOLDER,SPEED LEVER		1	
1-8	9004040120704	BOLT,M4x12L		2	
1-9	C12-001-01500	SPEED LEVER COMP POSITION		1	
1-10	9004030040504	BOLT,M3x4L		2	

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)

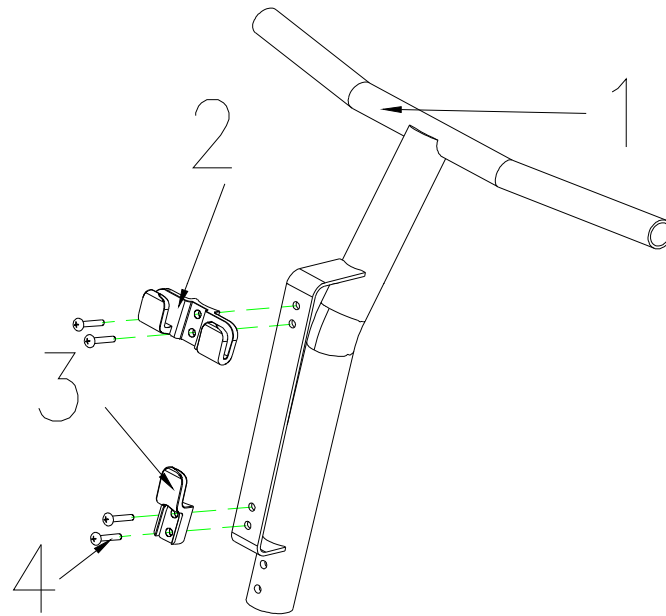


2. T TYPE HANDLE BAR ASS'Y

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)				
Fig&Index NO.	Parts No.	Parts Name	QTY	Remark
		English		
2	S02-055-00100	T TYPE HANDLE BAR ASS'Y	1set	
2-1	C02-055-00100	T TYPE HANDLE BAR	1	
2-2	C26-009-01601	BASKET HOLDER,PAINTED,UPPER	1	1904362
2-3	C26-009-01700	BASKET HOLDER,PAINTED,LOWER	1	1904363
2-4	9005060401003	BOLT,M6x35L	4	

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)

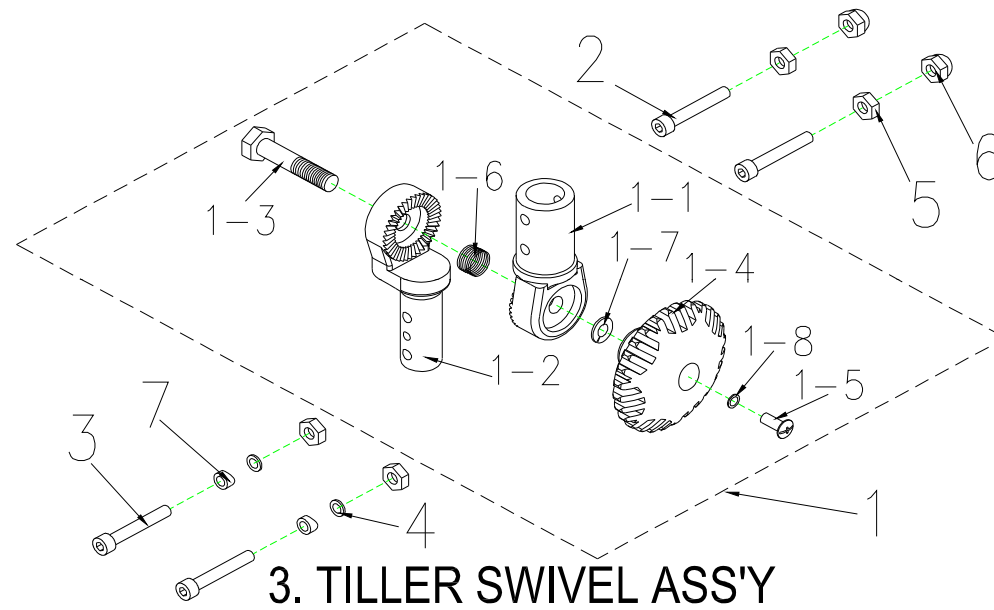


Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)				
Fig&Index NO.	Parts No.	Parts Name	QTY	Remark
		English		
3	S02-001-00501	TILLER SWIVEL ASS'Y	1set	
3-1	C02-009-01501	TILLER SWIVEL ASS'Y	1	
3-1-1	C02-009-00501	TILLER SWIVEL,UPPER	1	
3-1-2	C02-009-00603	TILLER SWIVEL,LOWER	1	
3-1-3	9036100501503	BOLT,TILLER,M10x50L	1	
3-1-4	C02-026-00100	NUT,TILLER SWIVEL	1	
3-1-5	9005050120802	CR ROUND HD SCREW,M5X12L	1	
3-1-6	C02-009-00700	SPRING,TILLER SWIVEL	1	
3-1-7	9211100200172	SPRING WASHER , M10	1	
3-1-8	9211050100802	SPRING WASHER , M5	1	
3-2	9035060451003	BOLT,M6X45	2	
3-3	9035060401003	BOLT,M6X40	2	
3-4	9250060100137	WASHER,M6	2	
3-5	9201060601003	LOCK NUT,M6	4	
3-6	9205060951003	CAP NUT,M6	2	
3-7	C02-009-01400	WASHER	2	

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)

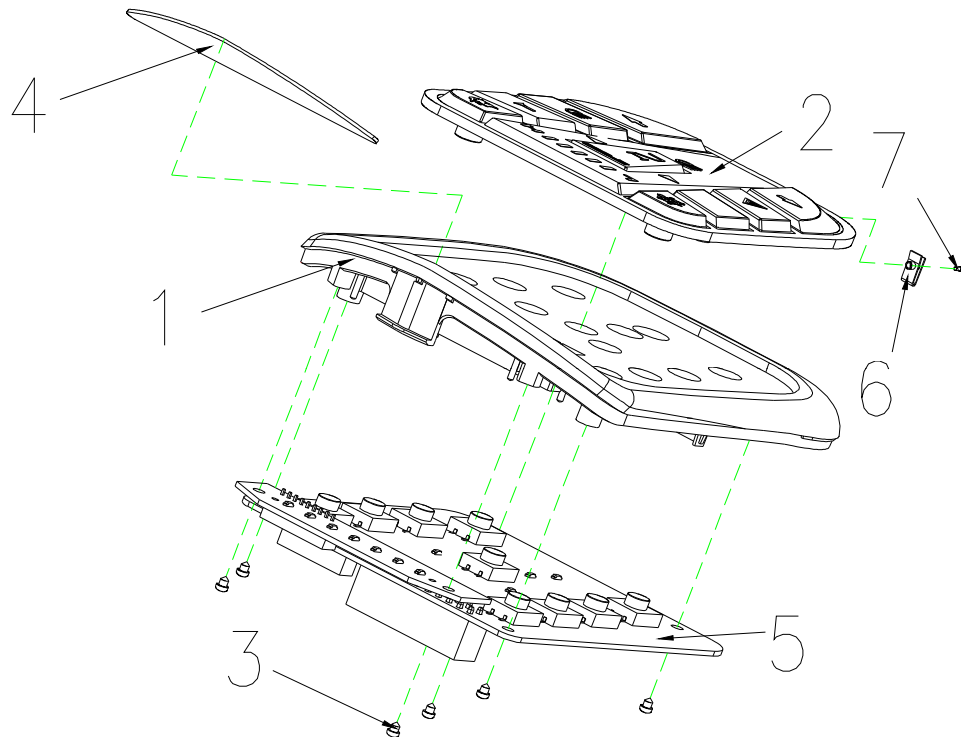


4. COVER ASS'Y , HANDLE BAR , UPPER

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)				
Fig&Index NO.	Parts No.	Parts Name	QTY	Remark
		English		
4	S08-054-00100	COVER , HANDLE BAR ASS'Y , UPPER	1set	1906055
4-1	C08-054-00600	COVER , HANDLE BAR , UPPER	1	
4-2	C10-051-00600	CONTROL PANEL BUTTON	1	1904369
4-3	9011030080502	BOLT M3x8L	6	
4-4	C27-054-00100	BATTERY INDICATOR	1	1904368
4-5	C10-051-00500	CONTROL PANEL PC BOARD	1	
4-6	9237040000003	U-TYPE SPRING NUT,M4	1	
4-7	9029040120002	BOLT M4x12L	1	

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)

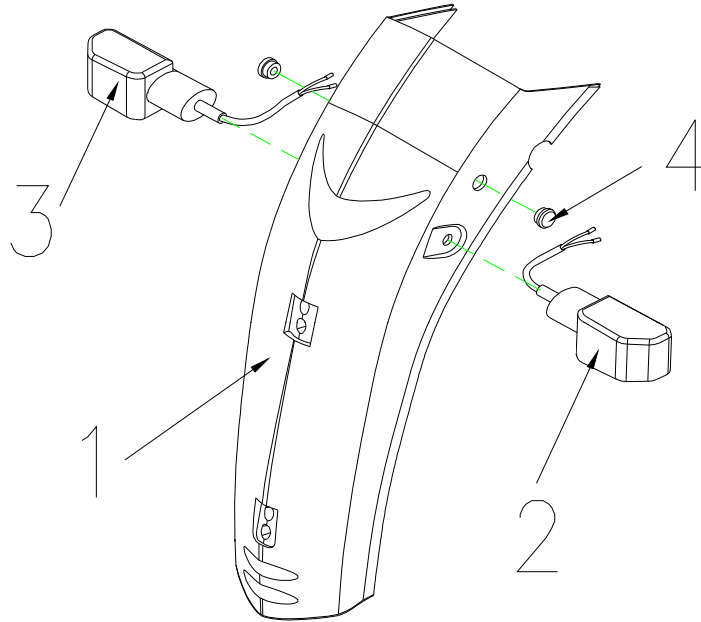


5. FRONT COVER ASS'Y

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)				
Fig&Index NO.	Parts No.	Parts Name	QTY	Remark
		English		
5	S08-055-00157	FRONT COVER ASS'Y , HAND BAR, BLUE 57 , W/O LABEL	1set	
5-1	C08-055-00257	FRONT COVER , HAND BAR, BLUE 57	1	1904376
5-2	C15-054-00100	FRONT SIGNAL LAMP , L	1	1904378
5-3	C15-054-00200	FRONT SIGNAL LAMP , R	1	1904379
5-4	C25-001-00300	PLUG TO COVER THE HOLE OF BRAKE	2	1904425
5	S08-055-00166	FRONT COVER ASS'Y , HAND BAR, BLACK 66 , W/O LABEL	1set	
5-1	C08-055-00266	FRONT COVER , HAND BAR, BLACK 66	1	1904377
5-2	C15-054-00100	FRONT SIGNAL LAMP , L	1	
5-3	C15-054-00200	FRONT SIGNAL LAMP , R	1	
5-4	C25-001-00300	PLUG TO COVER THE HOLE OF BRAKE	2	

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)

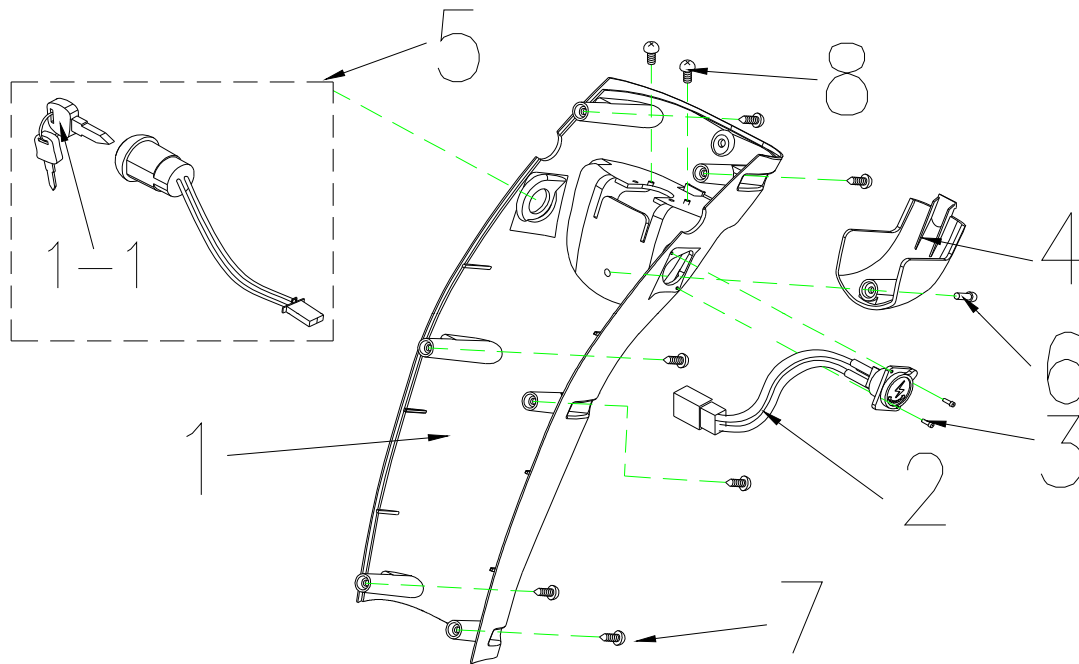


6. REAR COVER ASS'Y

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)				
Fig&Index NO.	Parts No.	Parts Name	QTY	Remark
		English		
6	S08-055-00257	REAR COVER ASS'Y , HAND BAR , BLUE 57	1set	
6-1	C08-055-00357	REAR COVER , HAND BAR , BLUE 57	1	1904380
6-2	C12-001-10202	CHARGER SOCKET ASS'Y	1	1904382
6-3	9014250080002	CR PAN HD SCREW , M2.5x8L	2	
6-4	C08-045-00457	DECORATED SHROUND FOR LEVER BASE, BLUE 57	1	1904453
6-5	C18-001-01500	IGNITION SWIITCH ASS'Y	1	
6-5-1	C37-001-02300	KEY	1	
6-6	9004040080704	BOLT,M4x8L	1	
6-7	9011040200003	BOLT,M4x20L	6	
6-8	9004030080504	BOLT,M3x8L	2	
6	S08-055-00266	REAR COVER ASS'Y , HAND BAR , BLACK 66	1set	
6-1	C08-055-00366	REAR COVER , HAND BAR , BLACK 66	1	1904381
6-2	C12-001-10202	CHARGER SOCKET ASS'Y	1	
6-3	9014250080002	CR PAN HD SCREW , M2.5x8L	2	
6-4	C08-045-00466	DECORATED SHROUND FOR LEVER BASE, BLACK 66	1	1904454
6-5	C18-001-01500	IGNITION SWIITCH ASS'Y	1	
6-5-1	C37-001-02300	KEY	1	
6-6	9004040080704	BOLT,M4x8L	1	
6-7	9011040200003	BOLT,M4x20L	6	
6-8	9004030080504	BOLT,M3x8L	2	

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)

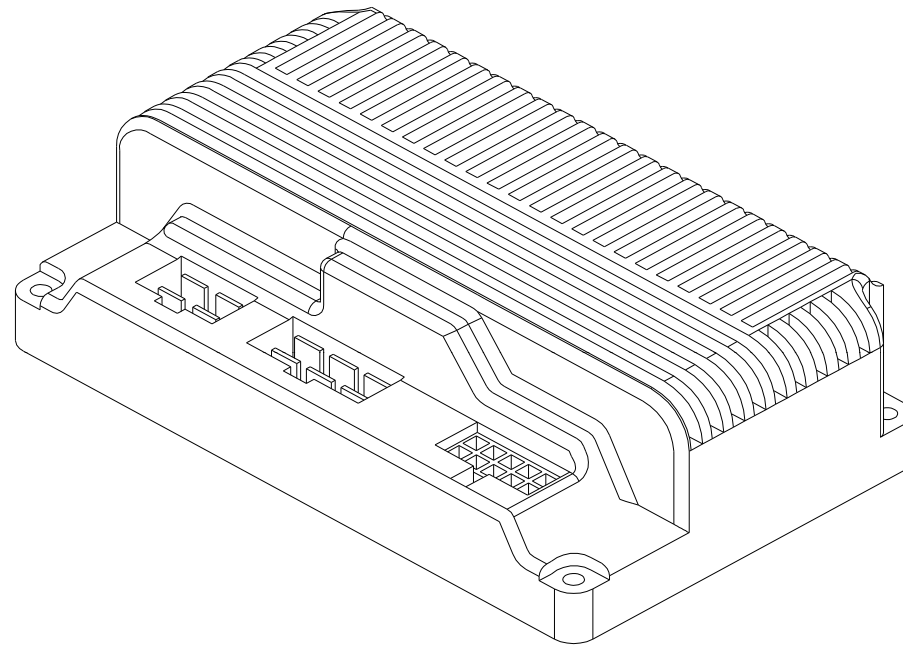


7.CONTROLLER , CTE 70A

Ceres 3 (M3B)


Ceres 3 (M3B) Parts List (2009/05/20)				
Fig&Index NO.	Parts No.	Parts Name	QTY	Remark
		English		
7	C10-001-01100	CONTROLLER , CTE 70A	1	1904383

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)

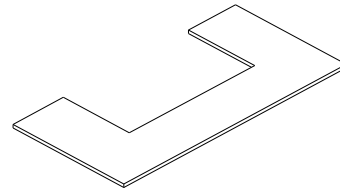

8.RUBBER PAD,BATTERY

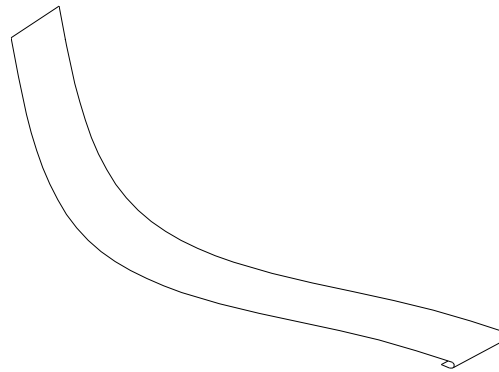


9.VELCRO STRIP,BATTERY

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)				
Fig&Index NO.	Parts No.	Parts Name	QTY	Remark
		English		
8	C25-003-00500	RUBBER PAD , BATTERY	1	
9	C07-001-90100	VELCRO STRIP , BATTERY	1	

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)

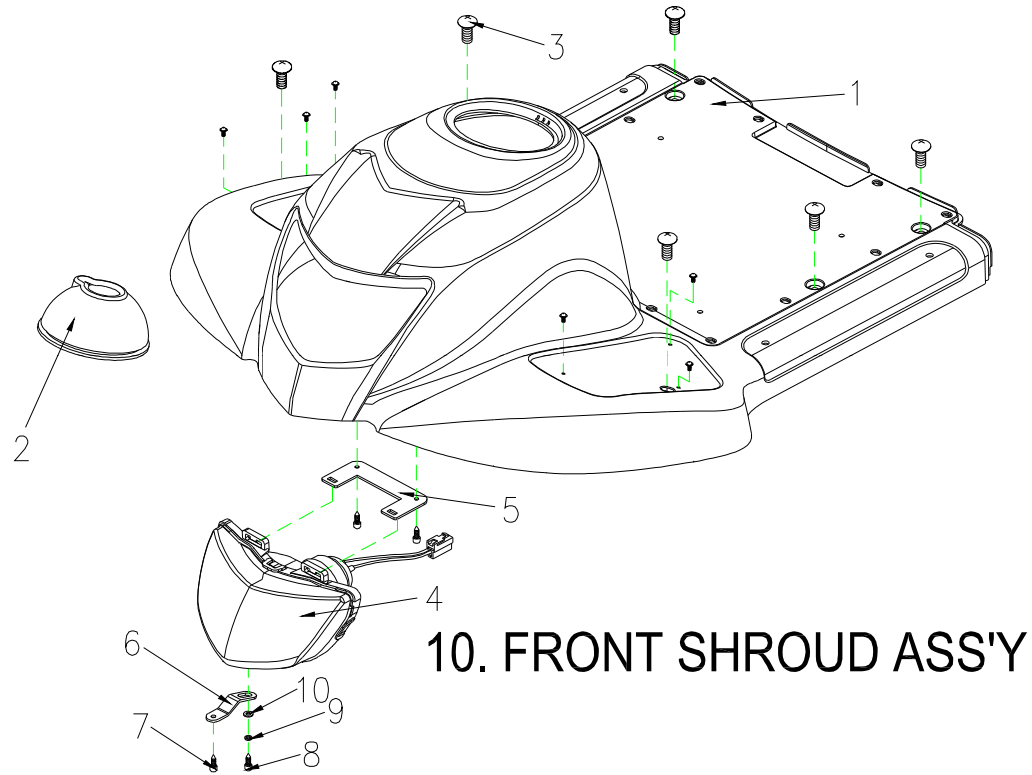


Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)				
Fig&Index NO.	Parts No.	Parts Name	QTY	Remark
		English		
10	S08-055-00357	FRONT SHROUD ASS'Y , BLUE 57	1set	
10-1	C08-055-00157	FRONT SHROUD , BLUE 57	1	1904445
10-2	C25-001-90101	RUBBER COVER , FRONT SHROUD	1	1904386
10-3	9005060201002	CR ROUND HD SCREW.M6X20	4	
10-4	C13-052-00100	HEADLIGHT	1	
10-5	C01-051-01300	HEADLIGHT BRACKET	1	
10-6	C01-051-01400	HEADLIGHT ADJUSTING PLATE	1	
10-7	9029040120002	BOLT M4x12L	9	
10-8	9005040120702	BOLT M4x12L	1	
10-9	9211050100803	SPRING WASHER , M5	1	
10-10	9214060100133	WASHER , M6	1	
10	S08-055-00366	FRONT SHROUD ASS'Y , BLACK 66	1set	
10-1	C08-055-00166	FRONT SHROUD , BLACK 66	1	1904444
10-2	C25-001-90101	RUBBER COVER , FRONT SHROUD	1	
10-3	9005060201002	CR ROUND HD SCREW.M6X20	4	
10-4	C13-052-00100	HEADLIGHT	1	1904263
10-5	C01-051-01300	HEADLIGHT BRACKET	1	1904264
10-6	C01-051-01400	HEADLIGHT ADJUSTING PLATE	1	1904265
10-7	9029040120002	BOLT M4x12L	9	
10-8	9005040120702	BOLT M4x12L	1	
10-9	9211050100803	SPRING WASHER , M5	1	
10-10	9214060100133	WASHER , M6	1	

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)

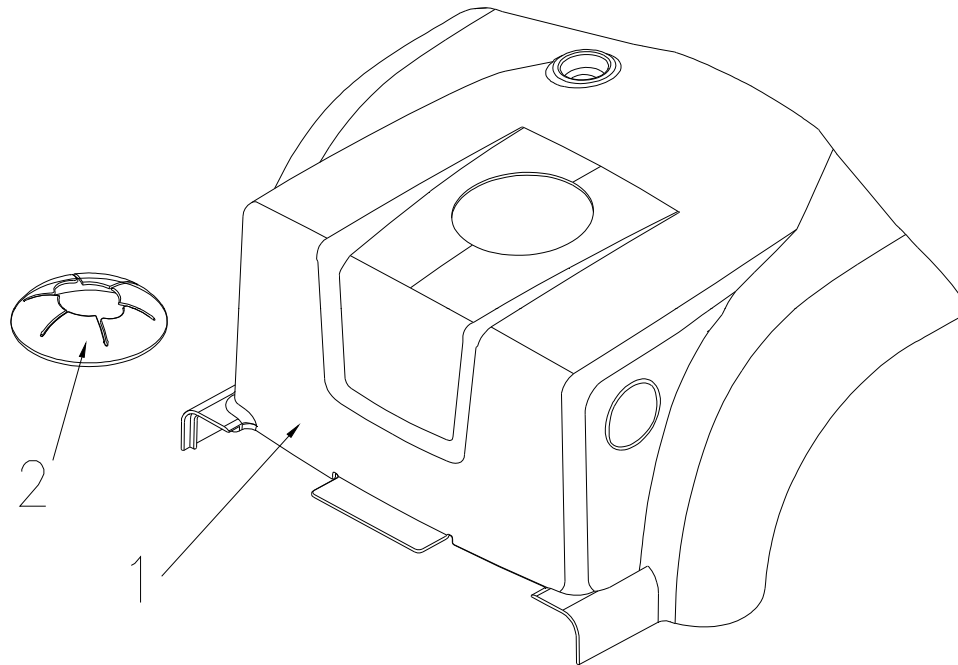


11.COVER BATTERY ASS'Y

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)				
Fig&Index NO.	Parts No.	Parts Name English	QTY	Remark
11	S08-054-00557	COVER ASS'Y , BATTERY , BLUE 57 , W/O LABEL	1set	
11-1	C08-054-00357	COVER BATTERY , BLUE 57	1	1904387
11-2	C25-039-00100	RUBBER COVER , REAR SHROUD	1	1904390
11	S08-054-00566	COVER ASS'Y , BATTERY , BLACK 66 , W/O LABEL	1set	
11-1	C08-054-00366	COVER BATTERY , BLACK 66	1	1904389
11-2	C25-039-00100	RUBBER COVER , REAR SHROUD	1	

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)

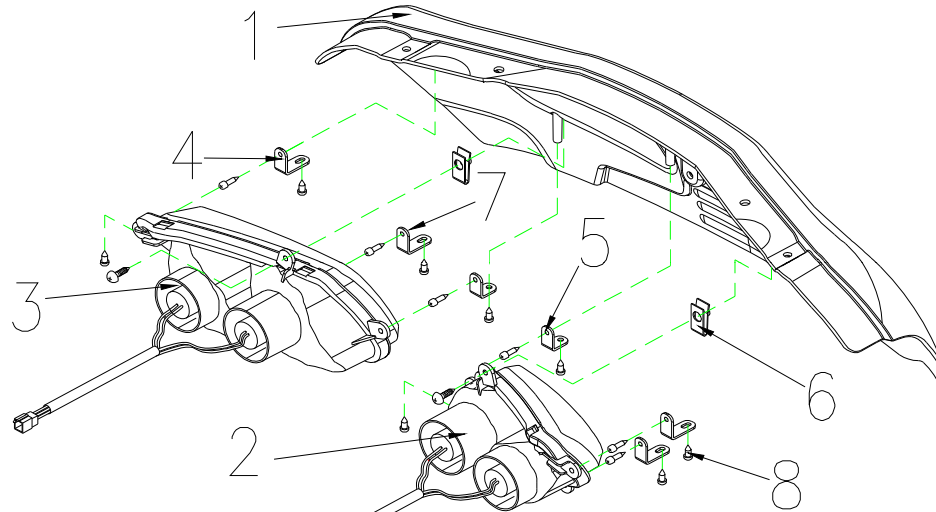


12.REAR SHROUD ASS'Y

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)				
Fig&Index NO.	Parts No.	Parts Name	QTY	Remark
		English		
12	S08-054-00657	REAR SHROUD ASS'Y , BLUE 57 , W/O LABEL	1set	
12-1	C08-054-00257	REAR SHROUD , BLUE 57	1	1904392
12-2	C14-052-00200	REAR SIGNAL LAMP , L	1	1904268
12-3	C14-052-00100	REAR SIGNAL LAMP , R	1	1904269
12-4	C01-051-01600	REAR SIGNAL LAMP BRACKET - LONG	4	
12-5	C01-051-01700	REAR SIGNAL LAMP BRACKET - SHORT	2	
12-6	9237040000003	U-TYPE SPRING NUT	2	
12-7	9004040100704	BOLT M4x10L	6	
12-8	9029040120002	BOLT M4x12L	8	
12	S08-054-00666	REAR SHROUD ASS'Y , BLACK 66 , W/O LABEL	1set	
12-1	C08-054-00266	REAR SHROUD , BLACK 66	1	1904393
12-2	C14-052-00200	REAR SIGNAL LAMP , L	1	
12-3	C14-052-00100	REAR SIGNAL LAMP , R	1	
12-4	C01-051-01600	REAR SIGNAL LAMP BRACKET - LONG	4	
12-5	C01-051-01700	REAR SIGNAL LAMP BRACKET - SHORT	2	
12-6	9237040000003	U-TYPE SPRING NUT	2	
12-7	9004040100704	BOLT M4x10L	6	
12-8	9029040120002	BOLT M4x12L	8	

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)

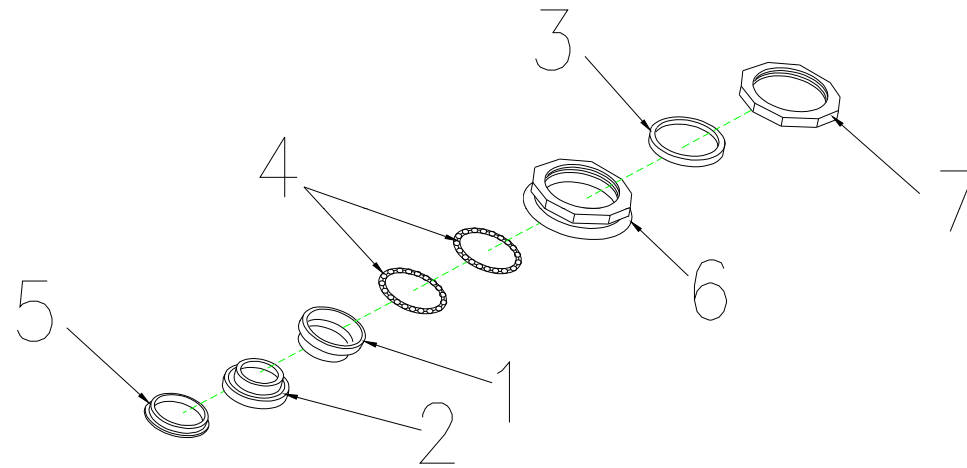


13.REGULAR BEARING BALL SET

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)					
Fig&Index NO.	Parts No.	Parts Name		QTY	Remark
		English			
13	S02-001-00601	REGULAR BEARING BALL SET		1set	
13-1	C29-001-10101	CUP , UPPER		1	
13-2	C29-001-00101	CUP , LOWER		1	
13-3	9214290300353	COLLARLESS WASHER		1	
13-4	C29-001-10200	BEARING		2	
13-5	C29-001-10301	BEARING CAGE		1	
13-6	C29-001-10401	BEARING BUSH		1	
13-7	C29-001-10501	LOCKING NUT , BEARING		1	

Ceres 3 (M3B) Parts List (2009/05/20)

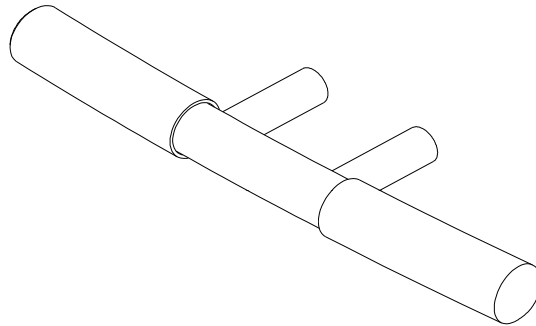


14.FRONT BUMPER

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)					
Fig&Index NO.	Parts No.	Parts Name		QTY	Remark
		English			
14	C01-055-00300	FRONT BUMPER		1	

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)

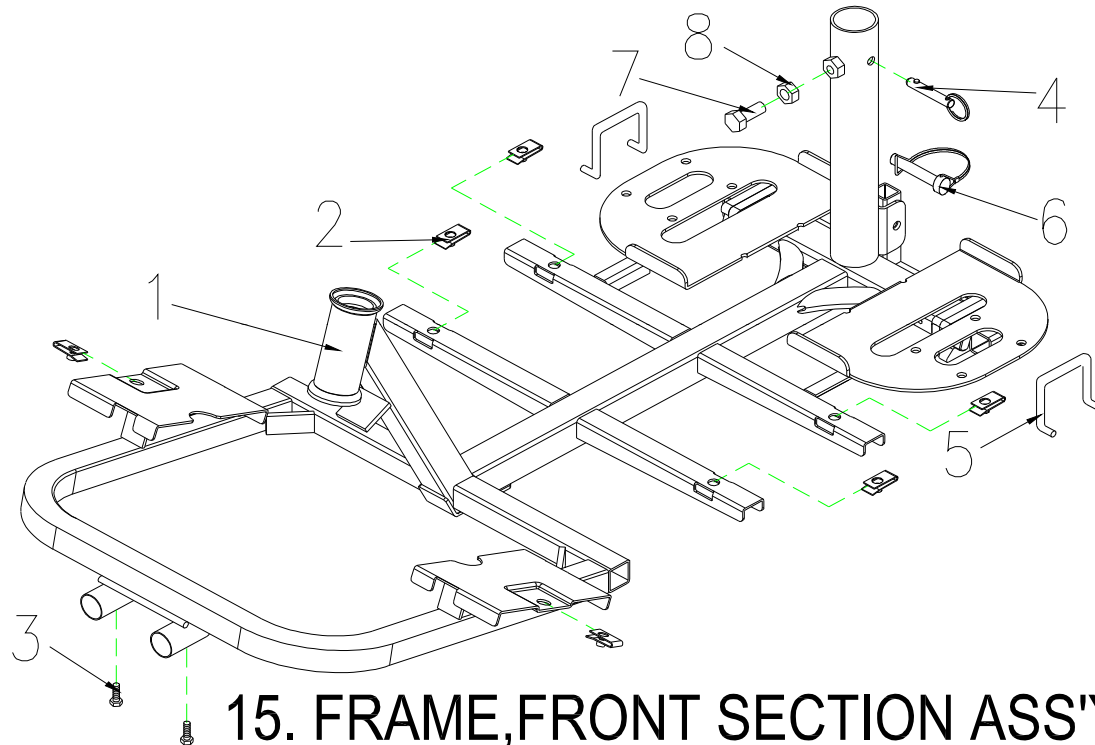


Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)					
Fig&Index NO.	Parts No.	Parts Name		QTY	Remark
		English			
15	S01-055-00100	FRAME,FRONT SECTION ASS'Y		1set	
15-1	C01-055-00100	FRAME,FRONT SECTION		1	
15-2	9237060000000	U-TYPE SPRING NUT,M6		6	
15-3	9002080301251	CR ROUND HD SCREW,M8x30L		2	
15-4	C01-001-10500	PIN,FRAME		1	
15-5	C01-003-01200	CLIP,VELCRO STRIP		2	
15-6	C01-005-00700	PIN LOCK , M8x41L		1	
15-7	9002100201251	CR ROUND HD SCREW,M10x20L		1	
15-8	9202100601252	NUT , M10x1.25		1	

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)

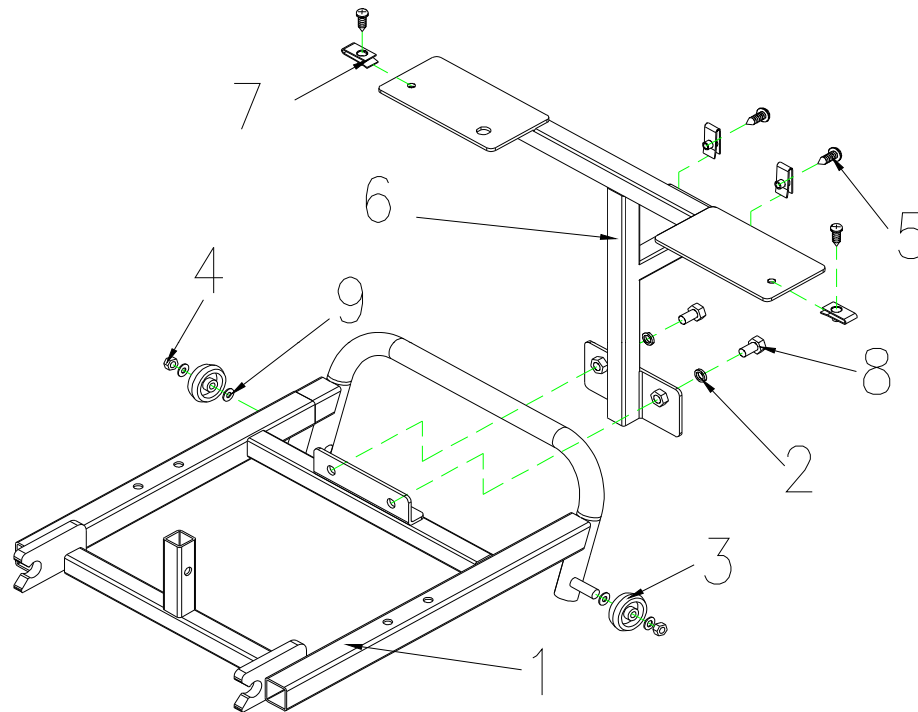


16. FRAME, REAR SECTION ASS'Y

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)				
Fig&Index NO.	Parts No.	Parts Name	QTY	Remark
		English		
16	S01-054-00200	FRAME,REAR SECTION ASS'Y	1set	
16-1	C01-054-00200	FRAME , REAR SECTION	1	
16-2	9211080200142	WASHER-5/16	2	
16-3	C28-007-00200	ANT-TIP WHEELS	2	
16-4	9201080801252	LOCK NUT , M8	2	
16-5	9005060201002	TAPPING SCREW , M6X15	4	
16-6	C01-054-00400	REAR BRACKET	1	
16-7	9237060000001	U-TYPE SPRING NUT,M6	4	
16-8	9001080201251	CR ROUND HD SCREW,M8x20L	2	
16-9	9214080150232	WASHER	4	

Ceres 3 (M3B)


Ceres 3 (M3B) Parts List (2009/05/20)

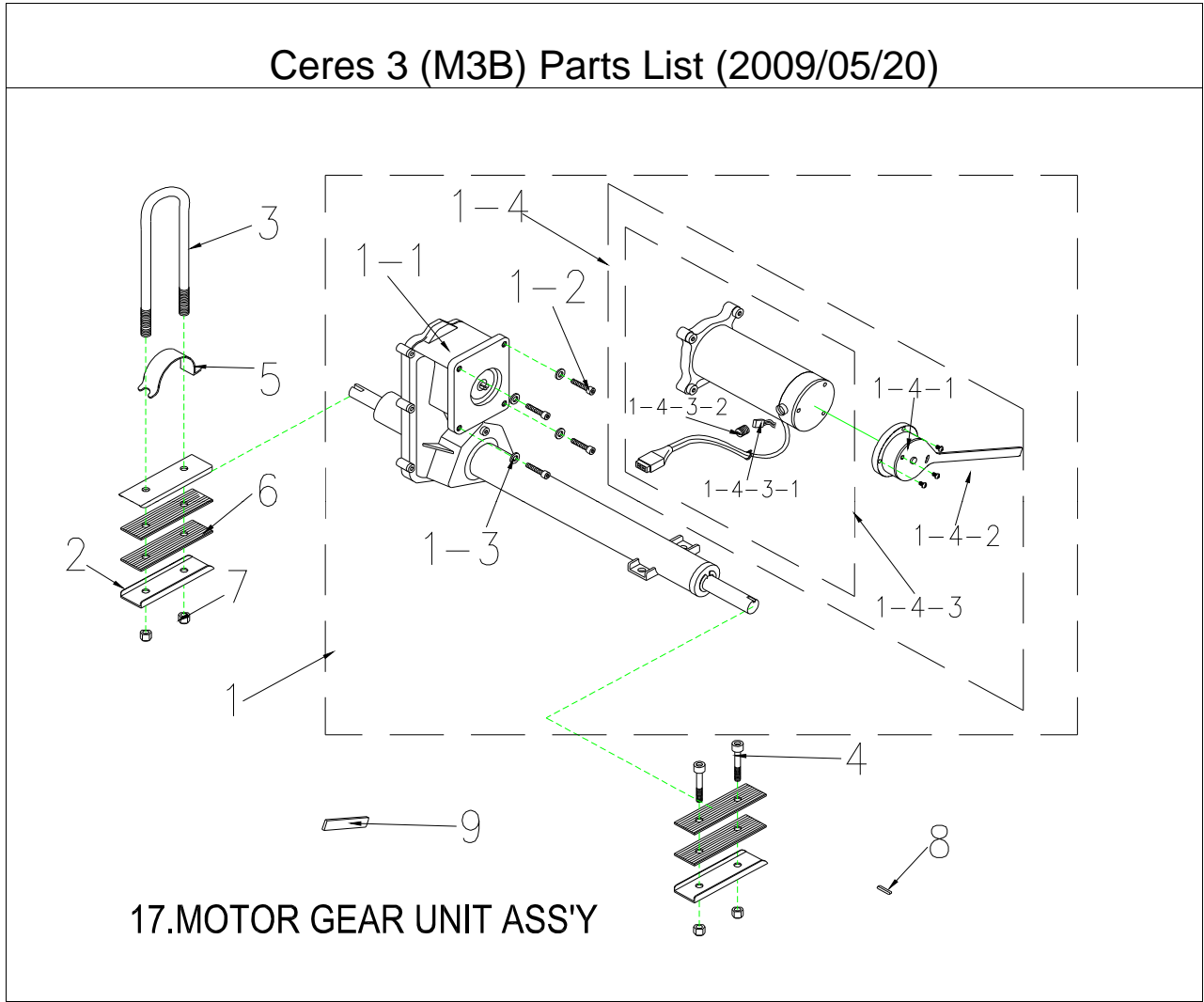

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)				
Fig&Index NO.	Parts No.	Parts Name	QTY	Remark
		English		
17	S09-054-00100	REAR DRIVE ASS'Y (W/I BOLT / NUT) ,9M/5300RPM/26:1	1set	
17-1	C09-054-00100	REAR DRIVE ASS'Y (W/O BOLT / NUT) ,9M/5300RPM/26:1	1	
17-1-1	C09-054-00200	TRANSAXLE ONLY (W/O MOTOR) 26:1	1	1904397
17-1-2	9021080201251	BOLT,M8x20	4	
17-1-3	9211080200142	SPRING WASHER 5/16	4	
17-1-4	C09-054-00300	MOTOR (W/I BRAKE) , 9M/5300RPM , 40KGF-CM	1	
17-1-4-1	C09-054-00400	ELECTRO-MAGNETIC BRAKE	1	1904398
17-1-4-2	9004040120704	BOLT,M4x12L	3	
17-1-4-3	C09-029-02400	MOTOR ONLY , 9M5300RPM	1	1904399
17-1-4-3-1	C09-005-00601	CARBON BRUSH , 7X11	2	1904400
17-1-4-3-2	C09-005-02400	CARBON BRUSH CAP , 7x11mm	2	1904401
17-2	C09-001-10500	U-TYPE PLATE,MOTOR	3	
17-3	9024080950001	U-TYPE BOLT,M8X1.25-98HX15MM	1	
17-4	9035080501251	BOLT,M8x50L	2	
17-5	C09-001-10400	ARCH TYPE PLATE,MOTOR	1	
17-6	C05-001-90200	RUBBER DAMPER PAD,MOTOR	4	
17-7	9201080801252	LOCK NUT,M8x1.25	4	
17-8	9357005005020	PIN,5x5x20	2	
17-9	C25-040-00100	GRIP	1	

Ceres 3 (M3B) Parts List (2009/05/20)

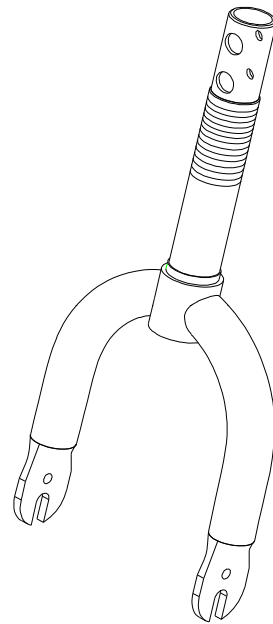


18. FRONT FORK ASS'Y

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)				
Fig&Index NO.	Parts No.	Parts Name	QTY	Remark
		English		
18	C03-042-00101	FRONT FORK ASS'Y	1	1904443

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)

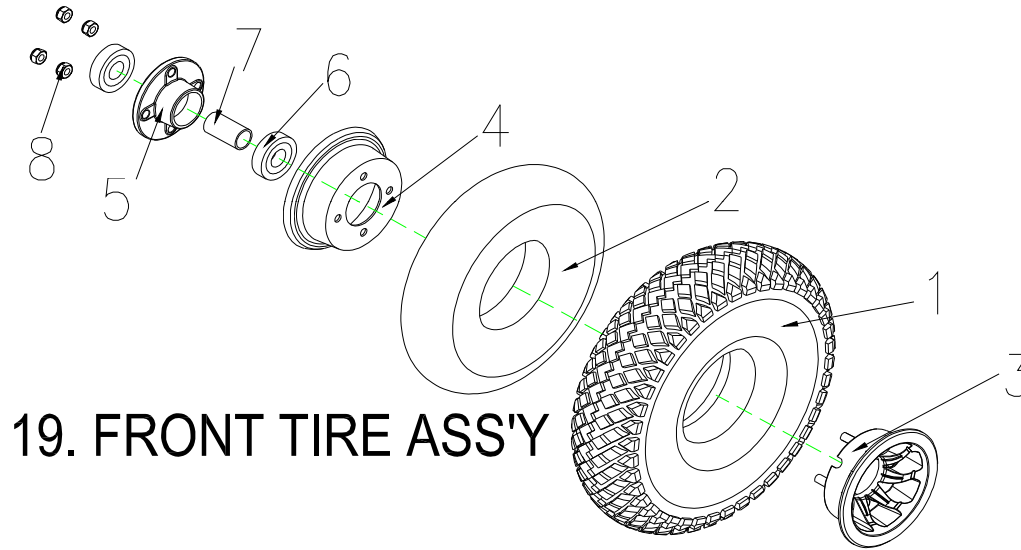


Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)				
Fig&Index NO.	Parts No.	Parts Name	QTY	Remark
		English		
19	S05-055-00100	FRONT TIRE ASSY,300-4-C154-1 GRAY	1set	1904407
19-1	C06-002-00600	TIRE , 300-4-C154 GRAY	1	1904440
19-2	C06-002-00300	INNER TUBE 300-4	1	1904439
19-3	C05-040-00100	WHEEL RIM , ALUNINUM , 4"	1	
19-4	C05-001-10400	WHEEL RIM , STEEL , 4"	1	
19-5	C05-001-10500	AXLE COLLAR,FRONT WHEEL	1	
19-6	9653063000000	BEARING	2	
19-7	C05-042-00200	SPACER , ψ 14X ψ 10.5X16L	1	
19-8	9201080801252	LOCK NUT,M8x1.25	4	

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)

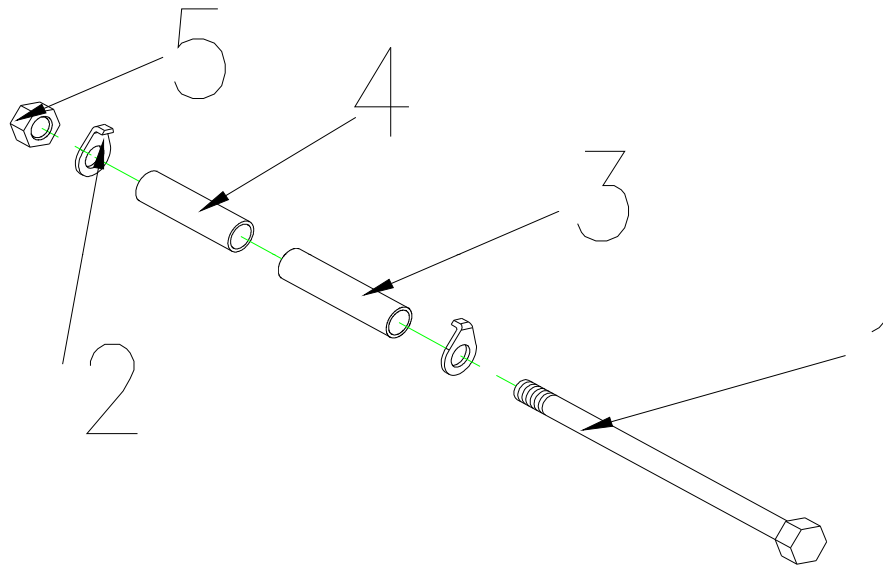


20.HEX FLANGE BOLT

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)					
Fig&Index NO.	Parts No.	Parts Name		QTY	Remark
		English			
20	S05-042-00100	HEX FLANGE BOLT , M10xP1.25x160L		1set	
20-1	9036101601251	BOLT		1	
20-2	9213100200192	FRONT FORK WASHER , M10		2	
20-3	C02-002-10200	SPACER-ID10.5mm.OD14mm.58L		1	
20-4	C02-002-10300	SPACER-ID10.5mm.OD14mm.34L		1	
20-5	9201101001252	LOCK NUT,M10x1.25		1	

Ceres 3 (M3B) Parts List (2009/05/20)

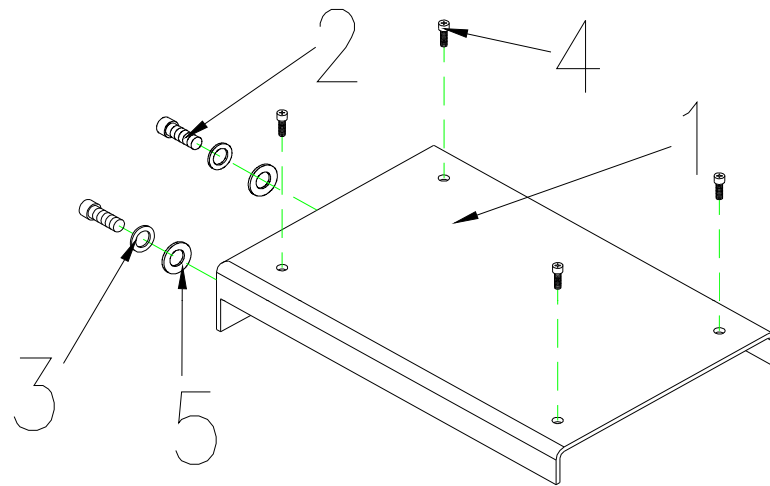


**21. CONTROLLER
HOLDER ASS'Y**

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)				
Fig&Index NO.	Parts No.	Parts Name	QTY	Remark
		English		
21	S01-054-00300	HOLDER , CONTROLLER ASS'Y	1set	
21-1	C01-054-00500	HOLDER , CONTROLLER	1	
21-2	9021080151251	HEX FLANGE BOLT , M8x15L	2	
21-3	9211080200142	SPRING WASHER-5/16	2	
21-4	9004040200703	BOLT , M4x20L	4	
21-5	9214080150232	WASHER 5/16	2	

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)

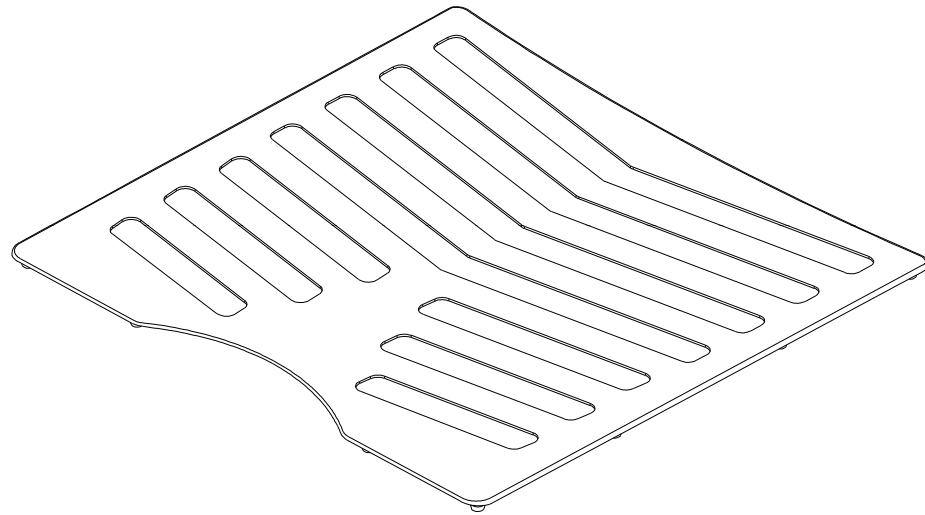


22. CARPET

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)					
Fig&Index NO.	Parts No.	Parts Name		QTY	Remark
		English			
22	C21-055-00100	CARPET		1	1904412

Ceres 3 (M3B) Parts List (2009/05/20)

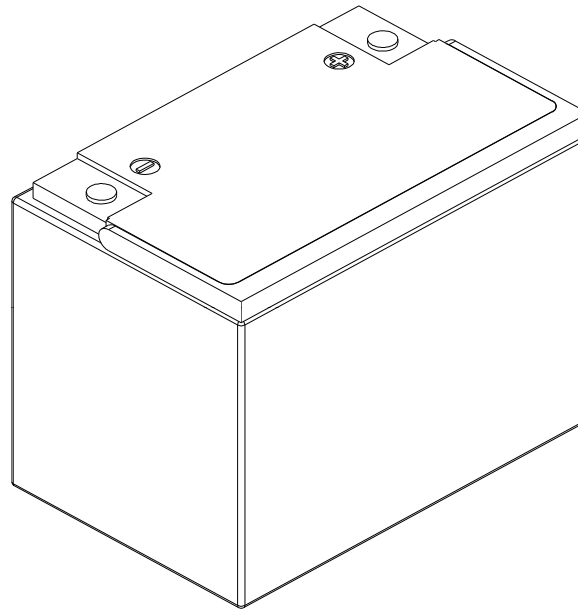


23. BATTERY U1

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)				
Fig&Index NO.	Parts No.	Parts Name	QTY	Remark
		English		
23	C17-001-23402	BATTERY , U1	1	

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)


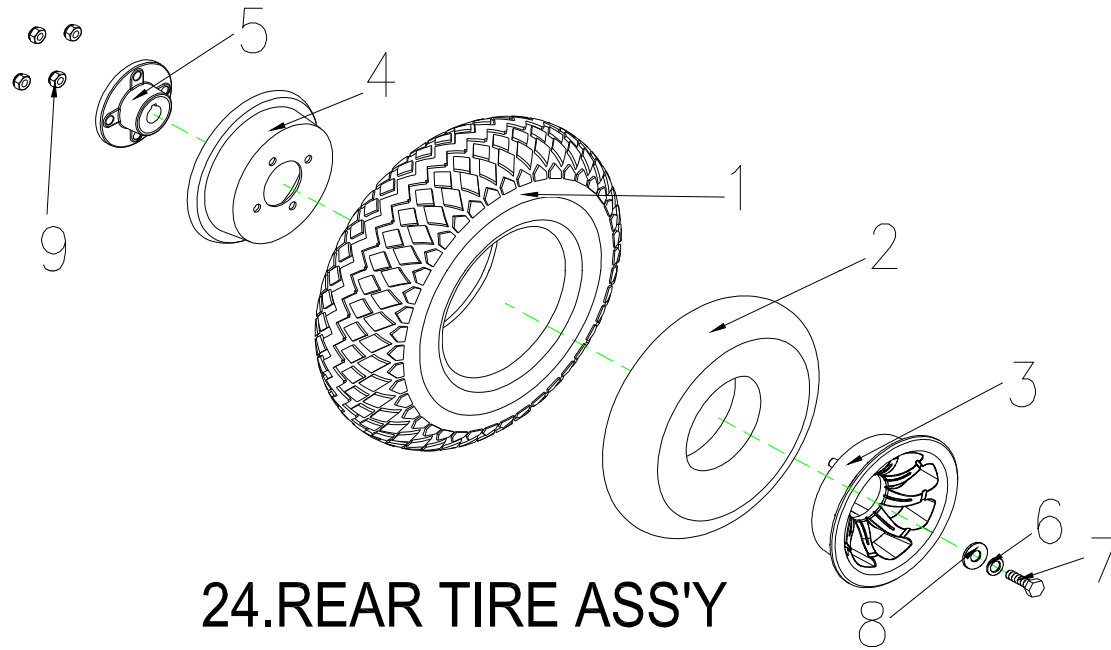


Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)				
Fig&Index NO.	Parts No.	Parts Name	QTY	Remark
		English		
24	S06-054-00100	REAR TIRE ASSY , GRAY , 400-5	1set	
24-1	C05-005-01400	TIRE , GRAY	1	1904442
24-2	C05-005-00200	INNER TUBE , 400-5	1	1904441
24-3	C06-029-00100	WHEEL RIM , ALUNINUM , 5"	1	
24-4	C05-005-00400	WHEEL RIM , STEEL , 5"	1	
24-5	C06-003-00502	AXLE COLLAR,REAR WHEEL	1	
24-6	9211080200142	SPRING WASHER,5/16	1	
24-7	9001080201251	BOLT,M8x20L	1	
24-8	9214080150232	WASHER,5/16	1	
24-9	9201080801252	LOCK NUT,M8x1.25	4	

Ceres 3 (M3B)


Ceres 3 (M3B) Parts List (2009/05/20)

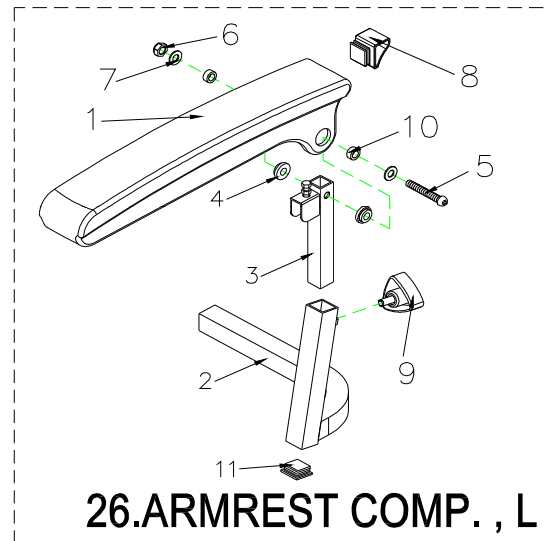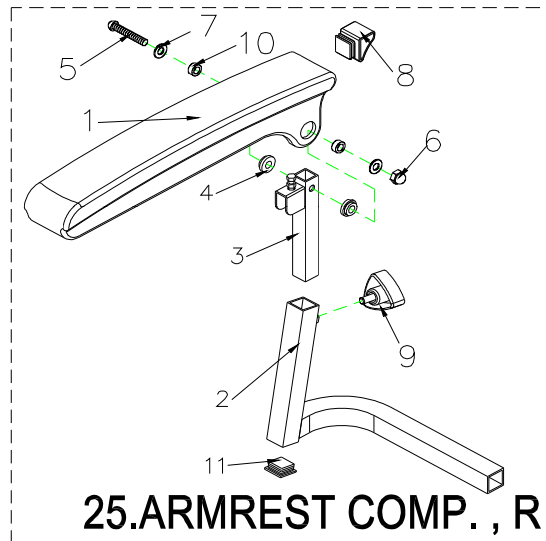

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)				
Fig&Index NO.	Parts No.	Parts Name	QTY	Remark
		English		
25	S07-053-00300	ARMREST COMP. , R	1set	
25-1	C07-044-00501	ARMREST , UPPER	1	
25-2	C07-053-00300	ARMREST , LOWER , R	1	
25-3	C07-053-00100	ARMREST , MIDDLE	1	
25-4	C07-044-01700	WASHER , 8mm	2	
25-5	9032080501253	BOLT , M8x50	1	
25-6	9206081251252	LOCK NUT , M8x1.25	1	
25-7	9214080100163	WASHER-M8	2	
25-8	C07-044-00600	PLUG	1	
25-9	C30-001-10201	BOLT,W/I KNOB , M8x20	1	
25-10	C05-017-00600	SPACER , φ 8.1X ψ 11X3T	2	
25-11	C26-001-00500	PLUG 1"X1"	1	
26	S07-053-00100	ARMREST , ADJUSTABLE , L	1set	
26-1	C07-044-00501	ARMREST , UPPER	1	
26-2	C07-053-00200	ARMREST ,LOWER , L	1	
26-3	C07-053-00100	ARMREST , MIDDLE	1	
26-4	C07-044-01700	WASHER , 8mm	2	
26-5	9032080501253	HEXAGON BOLT , M8x50(ED)	1	
26-6	9206081251252	LOCK NUT , M8x1.25	1	
26-7	9214080100163	WASHER , M8	2	
26-8	C07-044-00600	PLUG	1	
26-9	C30-001-10201	BOLT,W/I KNOB , M8x20	1	
26-10	C05-017-00600	SPACER , φ 8.1X ψ 11X3T	2	
26-11	C26-001-00500	PLUG 1"X1"	1	

Ceres 3 (M3B) Parts List (2009/05/20)

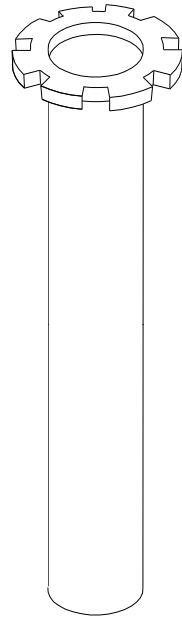


27. SEAT POST

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)				
Fig&Index NO.	Parts No.	Parts Name	QTY	Remark
		English		
27	C07-023-00102	SEAT POST	1	

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)

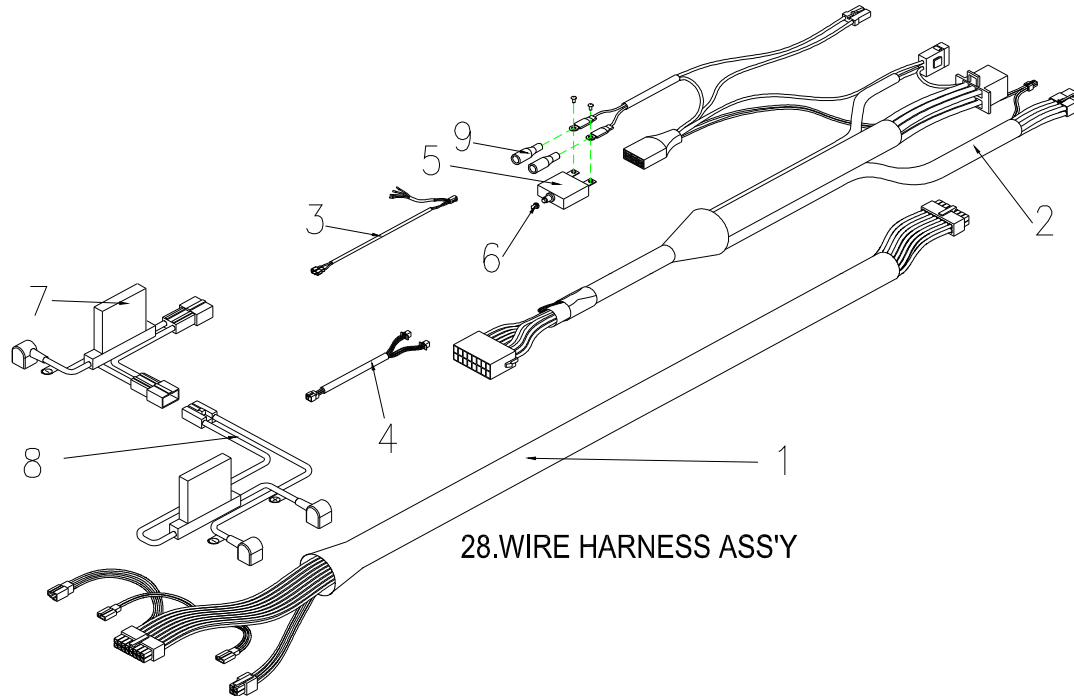


Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)				
Fig&Index NO.	Parts No.	Parts Name	QTY	Remark
		English		
28	S18-054-00100	WIRE HARNESS ASS'Y	1set	
28-1	C18-054-00100	WIRE HARNESS ASS'Y,FRONT	1	1904419
28-2	C18-054-00200	WIRE HARNESS ASS'Y,REAR	1	1904420
28-3	C18-054-00300	WIRE HARNESS FOR SIGNAL LIGHT	1	1904421
28-4	C18-051-00300	CABLE UNIT , REAR SHROUD	1	1904307
28-5	C16-001-10602	50A BREAKER,M11	1	1904424
28-6	C16-009-00400	TUBE,PROTECT SWITCH	1	
28-7	C18-001-10302	BATTERY CABLE , W/I FUSE (+)	1	1904426
28-8	C18-001-10402	BATTERY CABLE , W/I FUSE (-)	1	1904427
28-9	C26-001-00800	PLUG	2	

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)

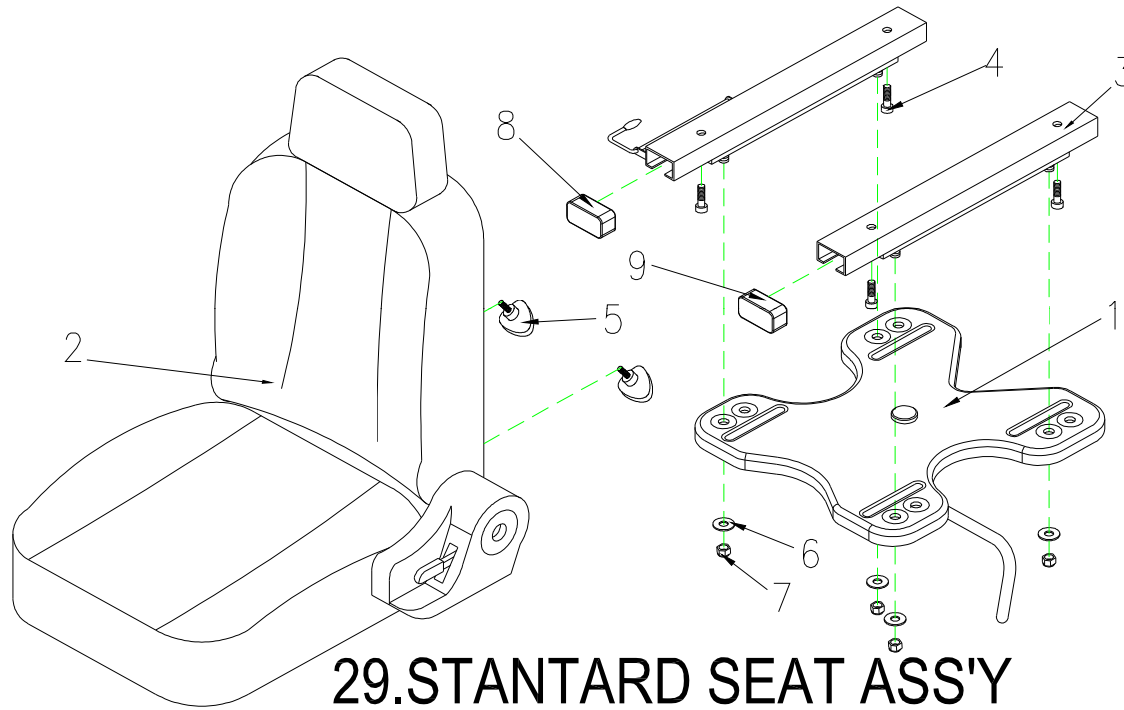

29.STANTARD SEAT ASS'Y

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)				
Fig&Index NO.	Parts No.	Parts Name	QTY	Remark
		English		
29	S07-055-00100	18" SEAT ASS'Y , BLACK , W/I SLIDER-100mm	1set	
29-1	C07-040-00800	STANTARD SEAT ASS'Y	1	
29-2	C07-051-00101	SEAT ASS'Y(SEAT PLATE/BLACK)	1	
29-3	C07-001-05800	SLIDER-100mm	1	
29-4	9032080101251	BOLT,M8X10L	4	
29-5	C30-001-10201	BOLT W/I KNOB ,M8X20L	2	
29-6	9214080150232	WASHER,5/16	4	
29-7	9201080801252	LOCK NUT,M8x1.25	4	
29-8	C26-003-00300	PLUG - R	1	
29-9	C26-003-00200	PLUG - L	1	
	C07-017-01900	HEADREST	1	

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)					
Fig&Index NO.	Parts No.	Parts Name		QTY	Remark
		English			
30		ACCESSORIES			
30-1	C20-007-00200	GRIP , 120mm		2	
30-2	C23-001-10300	BASKET		1	
30-3	C12-023-00100	CHARGER,220V, 4A OFF BOARD		1	
30-4	C12-001-00800	CABLE,CHARGER , EU		1	
30-5	C37-054-00100	OWNER'S MANUAL , BE 01		1	
30-6	C99-055-00100	CARTON , BE 01		1	
30-7	C37-001-00300	MIRROR , L		1	

Ceres 3 (M3B)

Ceres 3 (M3B) Parts List (2009/05/20)				
Fig&Index NO.	Parts No.	Parts Name	QTY	Remark
		English		
31		LABEL ACCESSORIES		
31-1	S27-054-00100	LABEL SET , REAR SHROUD	1set	
31-1-1	C27-001-01900	LABEL , NEVER LIFT UP	1	
31-1-2	C27-001-10700	LABEL , ELECTROMAGNETISM	1	
31-1-3	C27-001-00200	LABEL , CE	1	
31-1-4	C27-009-01700	LABEL , EMI	1	
31-1-5	C27-054-00200	LABEL , BRAND , BE 01	1	
31-1-6	C31-016-00200	VELCRO STRIP, LOOP , 80	1	
31-1-7	C31-016-00100	VELCRO STRIP, HOOK , 80	1	
31-1-8	C31-001-00200	VELCRO STRIP, LOOP , 45	1	
31-1-9	C31-001-00100	VELCRO STRIP, HOOK , 45	1	
31-1-10	C24-001-00300	REFLECTOR,SIDE,YELLOW	2	
31-2	S27-055-00100	LABEL SET , FRONT SHROUD	1set	
31-2-1	C26-047-00201	FOOTREST PAD , R	1	
31-2-2	C26-047-00101	FOOTREST PAD , L	1	
31-2-3	C08-029-00501	DAMAGE PROOF PIECE	2	
31-2-4	9906065000000	PIN ,M6.5	4	
31-2-5	C31-016-00200	VELCRO STRIP, LOOP , 80	1	
31-2-6	C31-016-00100	VELCRO STRIP, HOOK , 80	1	
31-3	S27-054-00300	LABEL SET , REAR SHROUD-BE 01	1set	
31-3-1	C27-034-01900	LABEL ,LOGO-BE 01	1	
31-3-2	C27-001-00801	LABEL , FREEWHEEL-ENGAGED	1	
31-3-3	C31-016-00200	VELCRO STRIP, LOOP , 80	1	
31-3-4	C31-016-00100	VELCRO STRIP, HOOK , 80	1	
31-3-5	C31-001-00200	VELCRO STRIP, LOOP , 45	1	
31-3-6	C31-001-00100	VELCRO STRIP, HOOK , 45	1	
31-3-7	C24-051-00100	REFLECTOR,REAR,RED	1	